

# M186

# Sakamoto Whole Body

# Pregnant Model



SAKAMOTO MODEL  
CORPORATION

**User's Manual (131025)**

— I N D E X —

- Introduction/Table of Contents 1
- Purpose/Content 2
- Cautions on Use 3
- Instructions for Use/Storage Instructions 4

<b>!</b> <b>Note</b>	<b>Confirm the main unit and accessories directly after the product arrives. If there is any defect or breakage, inform the dealer or Sakamoto Model.</b>
-------------------------	---

株式会社坂本モデル

Ⓢ

## Introduction

Thank you for purchasing our product, “Sakamoto Whole Body Pregnant Model.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

## Table of contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Cautions on Use.....	3
Instructions for Use／Storage Instructions.....	4


## Purpose

This product is used to perform the following training:

- 1) Rotating the fetus
- 2) Protecting the perineum
- 3) Checking the placenta and amnion
- 4) Free-style delivery

## Content

- 1) Obstetrics model unit (with abdominal cover, 1 genital model (skin color) and 4 screws) - 1 set
- 2) Replaceable genitalia (transparent) - Qty. 1
- 3) Vinyl fetus - Qty. 1
- 4) Placenta (with umbilical cord and amnion) - Qty. 1
- 5) Lubricant (hand soap) - Qty. 1
- 6) Powder - Qty. 1

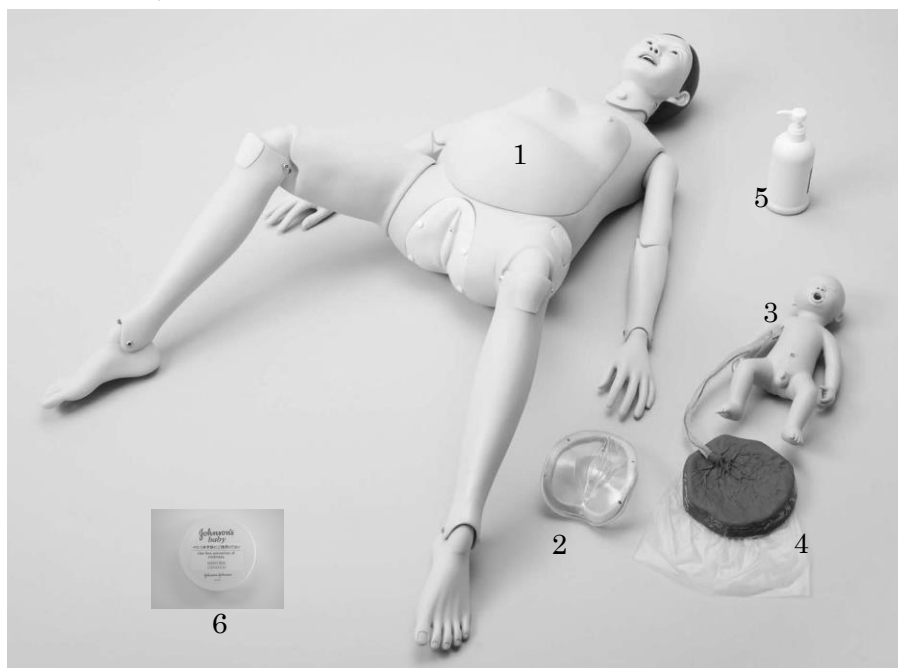

## **Cautions on Use**

- 1) If the resin made products are fitted onto the genitalia for long periods of time, the genitalia may deform. Therefore, handle with care when using them.
- 2) Always perform training in groups with a minimum of two people.
- 3) Be careful when handling the training product because the genitalia material gets dirty easily. In addition, be particularly careful with ballpoint pens and magic markers because their marks cannot be removed.
- 4) If the genitalia get slightly dirty, a neutral detergent can be used to clean it. (Do not use any solvents such as thinners, because the resin will dissolve.) Then, use a dry cloth to remove all remaining moisture and apply the powder.
- 5) The frame for the genitalia can be removed for washing, etc. However, always store the product with frame attached.
- 6) If any excessive load is applied or if dropped, it will damage the model.
- 7) When pushing out the fetus, always apply the lubricant first to ensure a smooth delivery.
- 8) After you finish training, thoroughly clean the lubricant that is on the fetus and genitalia.
- 9) When attaching the genitalia to the training model unit, make sure that the screw holes on the genitalia match the screw holes on the model before inserting the screws. If you force the screws when inserting them, it may damage the product.
- 10) If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 11) This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee the same results during actual treatment.
- 12) Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.

## **Instructions for Use**

1) Take out the training model unit from the hard case, and place it on a table with a non-slick surface in the desired position. (Abdominal cover is not used during delivery practice.)




2) Attach the hook on the placenta to the abdomen of the fetus.



3) Apply lubricant to the back of the fetus' head, shoulder and genitalia of the main unit.

4) The person in charge of the fetus is positioned at the upper body side of the training model and insert finger in mouth of the fetus. (This makes it easier to handle the fetus.)

5) The person in charge of the fetus places the fetus inside the main unit and pushes it out.

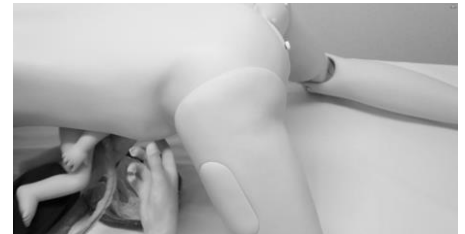

6) Begin training.

7) To replace the genitalia, turn the 4 screws that attach the genitalia to the training model in a counterclockwise direction, and remove the genitalia.

## **Storage Instructions**

1) First, thoroughly clean the lubricant that is stuck to the fetus and genitalia, and then dry out the pieces in the shade. If the genitalia become sticky, apply powder before storing.

2) Store the genitalia so that no excessive load is applied to it. Otherwise, the genitalia may become deformed.

3) Place the product inside the special case, and avoid storing the product in direct sunlight or in locations with high temperature and humidity.

---

**SAKAMOTO MODEL**  
C O R P O R A T I O N

---

**Head office**

34,Higashitakagicho,Shimogamo  
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

**Overseas Sales Department**

1-9-29,Senbahigashi,Minoh City,Osaka  
562-0035,JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com