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M212 Transparent Catheterization & Enema Model

M212-1 Transparent Female Catheterization & Enema Model

M212-2 Transparent male Catheterization & Enema Model

User’s Manual

[M212-1]
Transparent Female Catheterization & Enema Model

[M212-2]
Transparent male Catheterization & Enema Model

**Only use water* for training with
this product!**

* Please use tap water.

Table of contents

1	Introduction	28	6	Preparation	35
2	Product contents.....	29	7	How to use	41
3	Part names	30	8	Putting away	46
4	Cautions on use	31	9	Cleaning and storage ...	49
5	Product applications	34	10	Specifications	51

(Refer to the following page for a detailed Table of Contents.)

SAKAMOTO MODEL
C O R P O R A T I O N

Table of contents

1	Introduction	28
2	Product contents	29
3	Part names	30
4	Cautions on use	31
5	Product applications	34
6	Preparation	35
6-1	Overall Connection Diagram	35
6-2	How to attach the bladder and rectum parts	36
6-3	Preparation	38
7	How to use	41
7-1	Catheterization training	41
7-2	Enema training	44
7-3	Cleansing training	45
8	Putting away	46
9	Cleaning and storage	52
10	Specifications	51

1 Introduction

Thank you very much for purchasing the “Transparent Catheterization & Enema Model”.

This user’s manual contains cautions and notes on how to use this product safely. Please read this manual carefully in order to ensure that you know how to use the product safely. In addition, after reading the manual, keep it in a safe place where it can be referred to at any time later on.

After purchasing the product, check the package contents carefully to make sure that nothing is missing or broken. If you notice any problems, contact Sakamoto Model.

About this user’s manual

● Cautions and other symbols

This manual uses the following symbols to indicate cautions on handling and other information which may be useful to know during use.

	This indicates a caution that applies when using the product. If the instructions listed are not followed, personal injury or damage to the product or other equipment may result.
--	--

	This indicates information which may be useful to know during use, or which should be taken note of in order to use the product properly.
---	---

● About illustrations

For reasons such as product improvements, the illustrations which appear in this manual may differ in some parts from the product which you have. The details in the instructions will not have changed, so please follow the instructions.

● About the products covered in the explanation

The explanations in this document use [M212-1] Transparent Female Catheterization & Enema Model as an example, but please handle the Transparent Male Catheterization & Enema Model [M212-2] in the same way.

Guide to product’s introductory page

You can view this product’s introductory page on our company’s website by using either the following URL or the QR code*.

● Sakamoto Model Corporation’s Introductory page for the “M212 Transparent Catheterization & Enema Model”

<https://www.sakamoto-model.co.jp/excretion/m212/>


* QR code is a registered trademark of Denso Wave Inc.

2 Product contents


If no quantity is listed, the quantity included is one.

■ [M212-1] _____

■ [M212-2] _____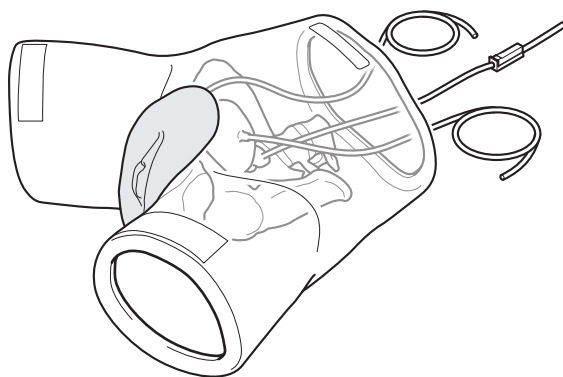


Female model body

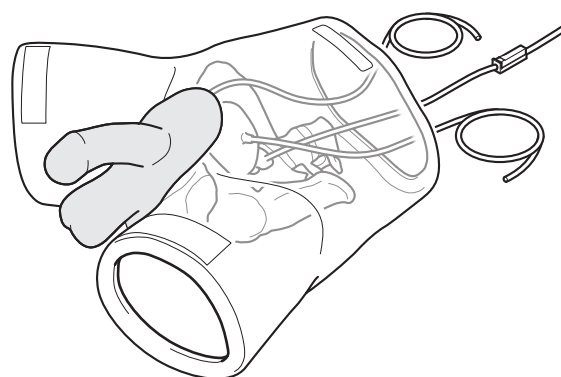Male model body

■ [M212-1] [M212-2] common _____

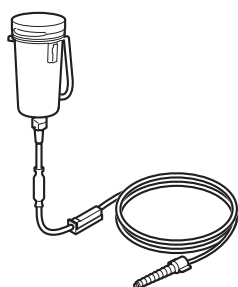Irrigator bottle


Irrigator bottle stand


Nelaton catheter
(8 EG)

Balloon catheter
(16 Fr)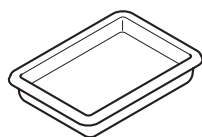


Tray

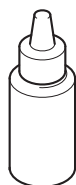Lubricant oil

Baby powder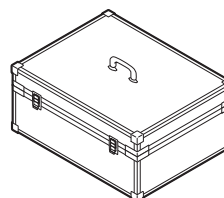


Exclusive case

● TIP


Syringe and enema are not included with this product.

3 Part names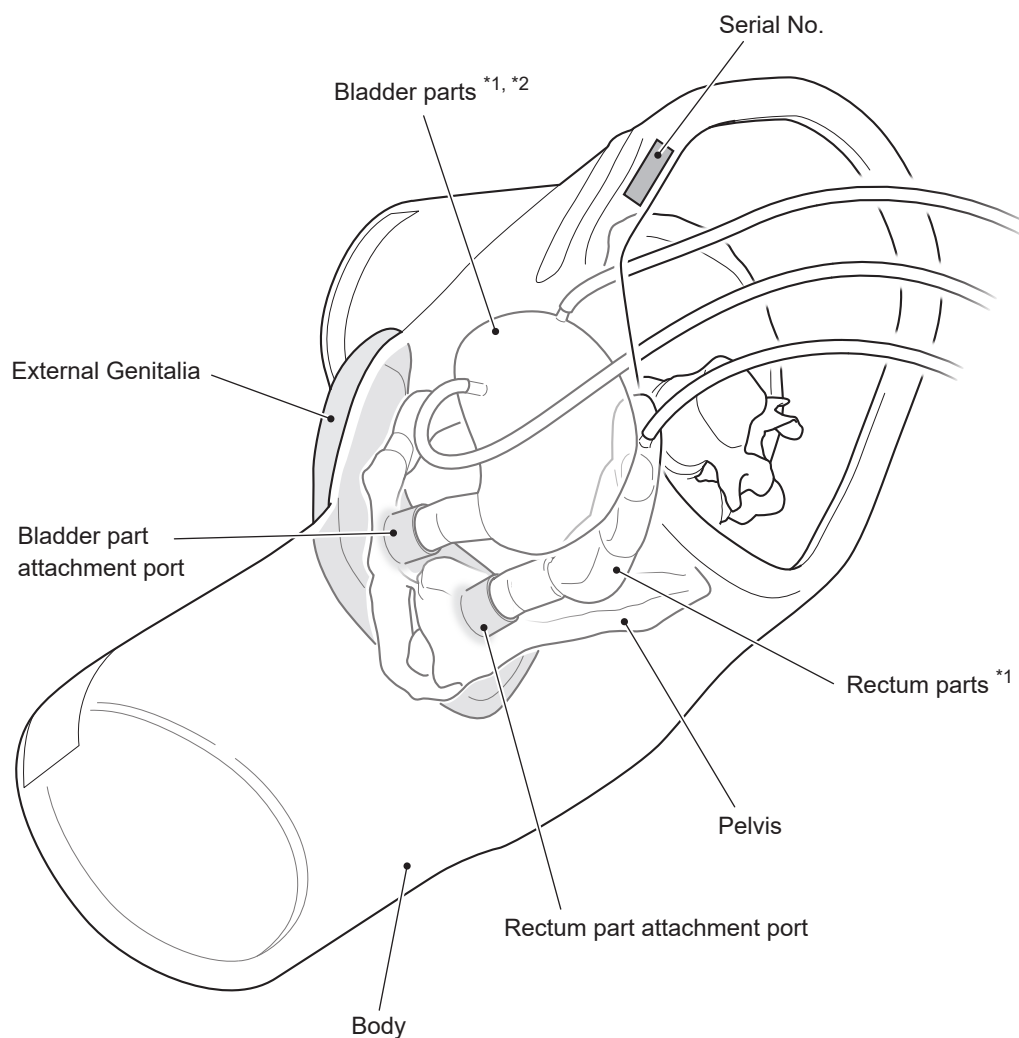


External Genitalia

<Female model body>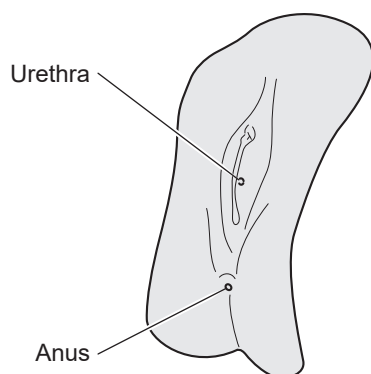

<Male model body>

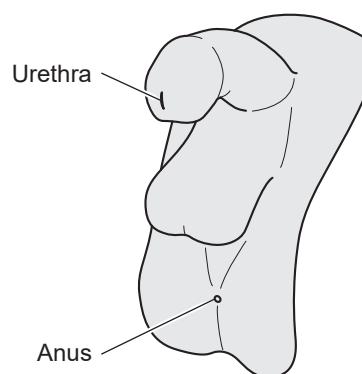*1 At the time of shipment, the bladder and rectum parts are already attached to the model main unit.

*2 The bladder part of the male model body depicts the prostate.

4 Cautions on use

In order to use this product safely, be sure to observe the following.
(Items which are particularly important are marked with “ ■ ”.)

■ External Genitalia

● **Wear gloves when handling the external genitalia.**

Wear gloves to prevent dust and dirt from sticking.

● **Handle these items gently.**

- Do not touch them with sharp objects such as knives, scissors or needles.
- Do not pull or twist them with a strong force.
- Do not subject them to severe shocks.

● **Do not write on the external genitalia with items such as ballpoint pens.**

If you write on the external genitalia using a ballpoint pen or a similar object, the surface of the body will become smeared with ink which cannot be removed.

● **Do not remove the external genitalia from the model.**

If the external genitalia comes off the model body, please do not try to put it back on yourself and contact Sakamoto Model.

● **Do not let it come into contact with other resin products.**

If other resin products come into contact with the external genitalia for a long time, the external genitalia may become deformed.

● **Do not touch against printed materials.**

If the body is placed on top of or wrapped in printed items such as newspapers and books, the body may become imprinted with ink which cannot be removed.

● **Wipe off dirt on the surface of the external genitalia using a neutral detergent.**

- If it is extremely dirty, wipe it with a cloth which has been moistened with diluted detergent, and then wipe it again with a dry cloth.
- Do not use benzine, thinner or other solvents. Otherwise, it may damage the product or cause the epidermis to deteriorate.
- After removing dirt, wipe off moisture and coat it with the accessory baby powder.

● **Apply attached baby powder to protect the external genitalia part and prevent stickiness.**

When not using the product for a long period of time or if the external genitalia is sticky, coat it with the accessory baby powder.

■ Body, Bladder parts, Rectum parts

● Handle with care

- Handle the body, bladder, and rectum parts with care.
- The body, bladder, and rectum parts are easily scratched. Hitting them against objects or rubbing them too hard can cause damage or clouding.

● Do not remove the tubes from the bladder or rectum parts.

Removing the tubes may damage them or cause water to leak from the connected parts when you reattach them.

● Do not use if the body is damaged.

If the body is damaged, discontinue use and contact Sakamoto Model. There is a risk of injury if you touch the damaged area.

■ General product handling

● Do not subject the product to impacts or other forces.

- Do not drop.
- Do not bump against solid objects.
- Do not apply excessive loads.

If any of the above occur, damage may result.

● Do not disassemble or modify the product.

If you attempt to disassemble or modify the product, it may become damaged or problems with operation may result.

● Please use only water as the liquid.

Using any liquid other than water may cause the product to deteriorate or emit an unpleasant odor.

● Do not use iodine-based chemicals, thinner, benzene, etc.

Using iodine-based chemicals, thinner, benzene, etc. will lead to product deterioration. Never use it.

● Do not use any lubricant oil other than the included lubricant oil.

Be sure to use the supplied lubricant oil. Using a lubricant oil other than the one provided may cause damage or deterioration of the external genitalia.

● If you notice any problems when using the product, stop using it.

If problems occur with the product during use, stop using it and contact Sakamoto Model. If you continue using the product, accidents may occur.

- **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, discoloration or deterioration may occur.

- **There are limits to the effectiveness of simulator training.**

This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee that training using this product will provide the same level of expertise as actual treatment.

- **Do not use this product for other applications.**

Do not use this product for applications other than the original application for which it is designed. The product may become damaged or injury may result.

5 Product applications

“Transparent Catheterization & Enema Model” is a transparent model that allows you to learn and simulate the anatomy related to urinary catheterization and enema.

■ Catheterization training

- Intermittent catheterization
- Continuous catheterization
- Checking the placement of the balloon

■ Enema training

- Glycerin enema

■ Cleansing training


- Genital washing

6 Preparation

6-1 Overall Connection Diagram

The following outline diagram shows the product with the various parts connected in preparation for use.

Refer to this diagram while carrying out the steps in “6-3 Preparation” (P.39 - 41).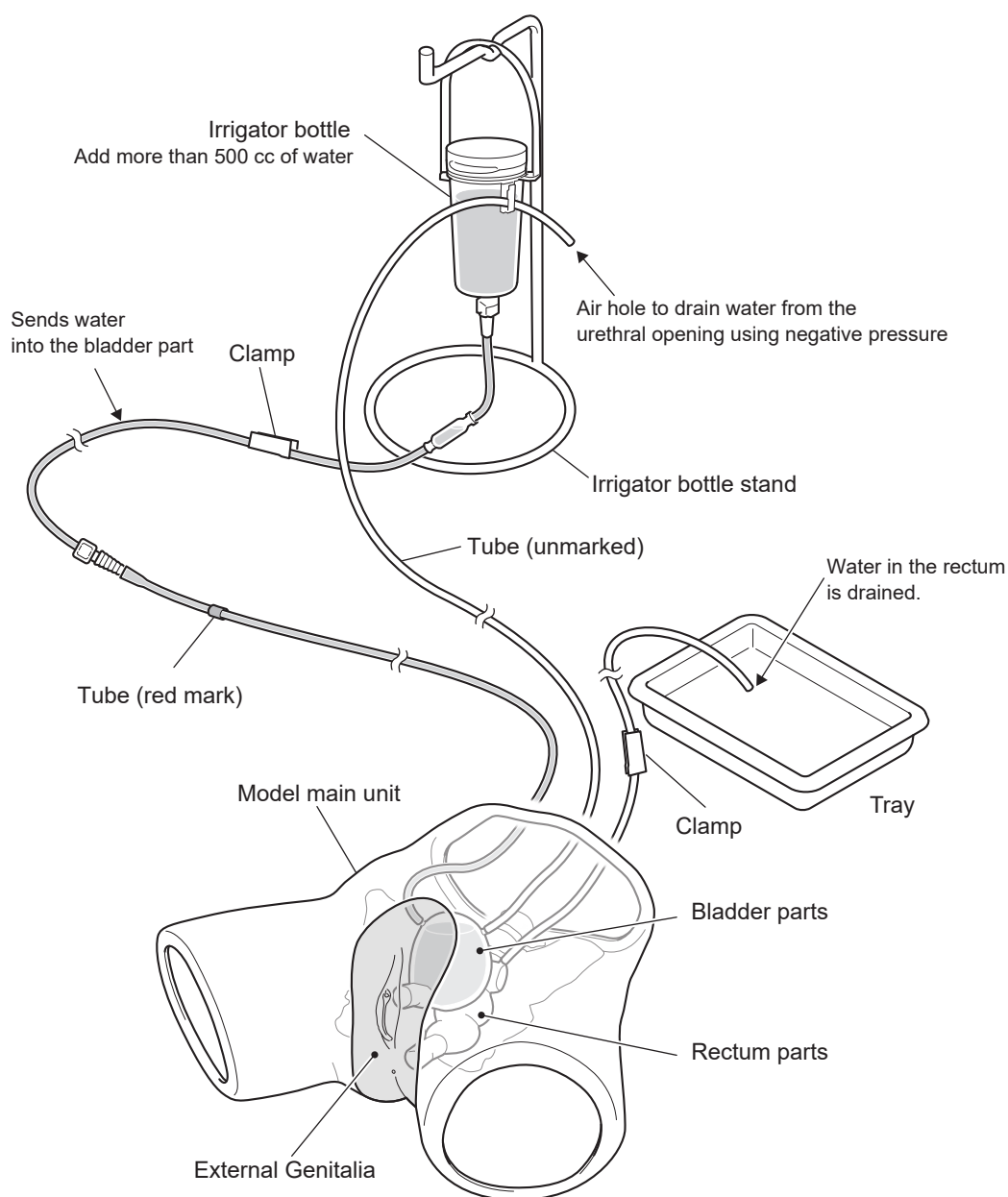


6-2 How to attach the bladder and rectum parts


At the time of shipment the bladder and rectum parts are already attached to the model main unit, but you will need to attach them yourself for the second and subsequent training sessions. Follow the steps below to attach the bladder and rectum parts.

CAUTION



- When attaching the bladder and rectum parts, do not hold the external genitalia. Pressing too hard or for a long period of time on the external genitalia may cause them to become deformed.
- Insert the bladder and rectum parts into the mounting ports until the marked lines are hidden.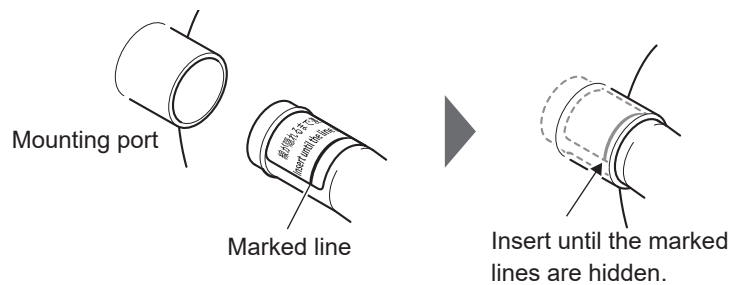

- 1** While supporting the model main unit in the position shown in the diagram with your hand, attach the bladder part to the bladder part attachment port.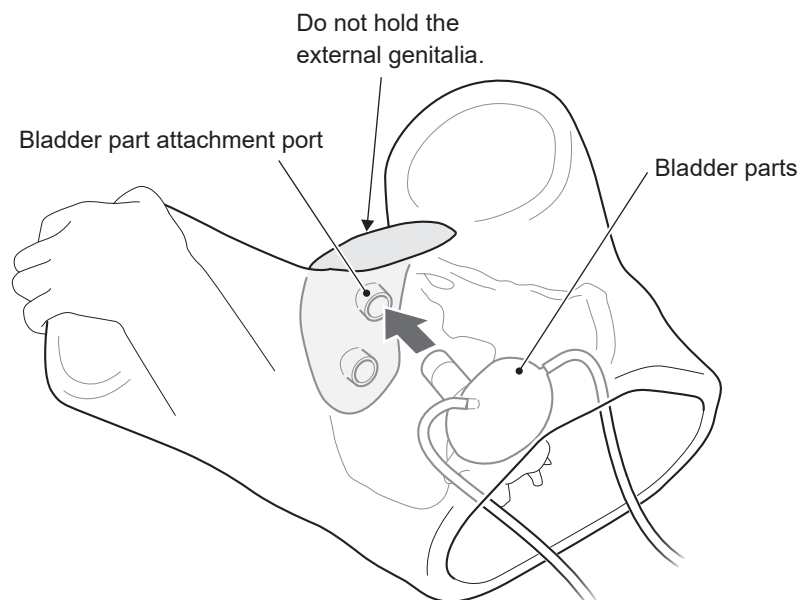


- 2** Insert the rectum part through the gap in the pelvis and attach it to the rectum part attachment port.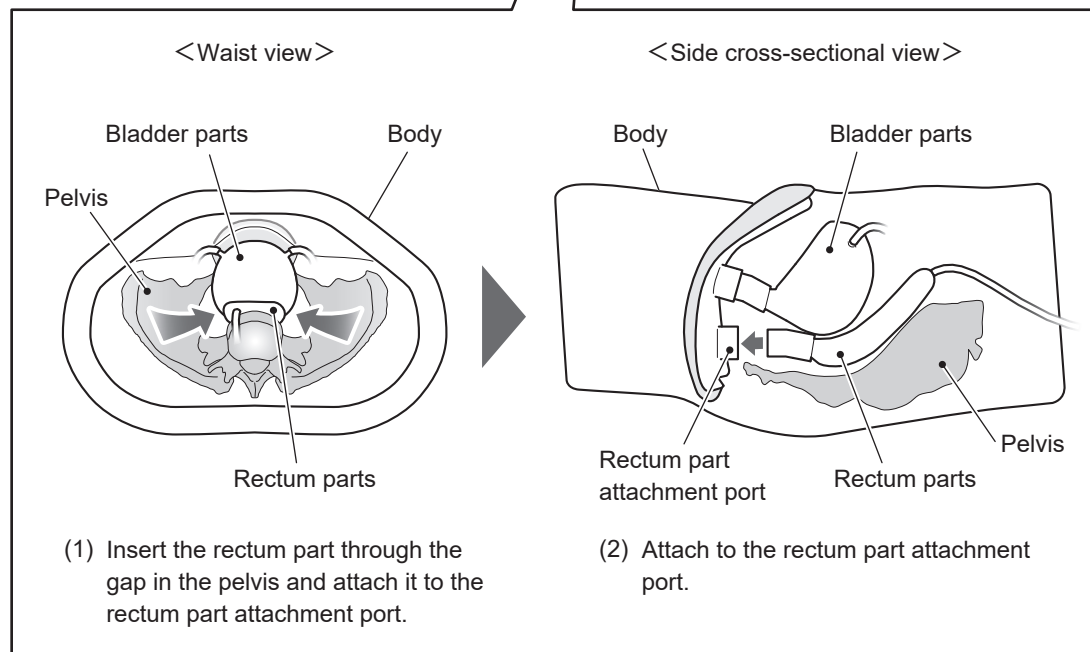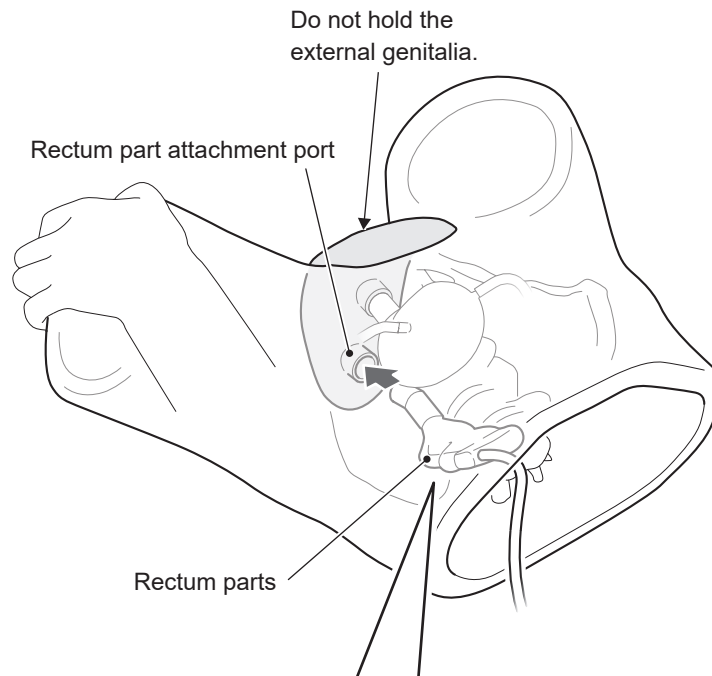


6-3 Preparation

To properly inject water into the bladder part, place the model main unit and the irrigator bottle stand on the same flat surface, such as a training desk.

- 1 Hang the irrigator bottle on the irrigator bottle stand and connect the tube (red mark) of the bladder part to the irrigator bottle tube.**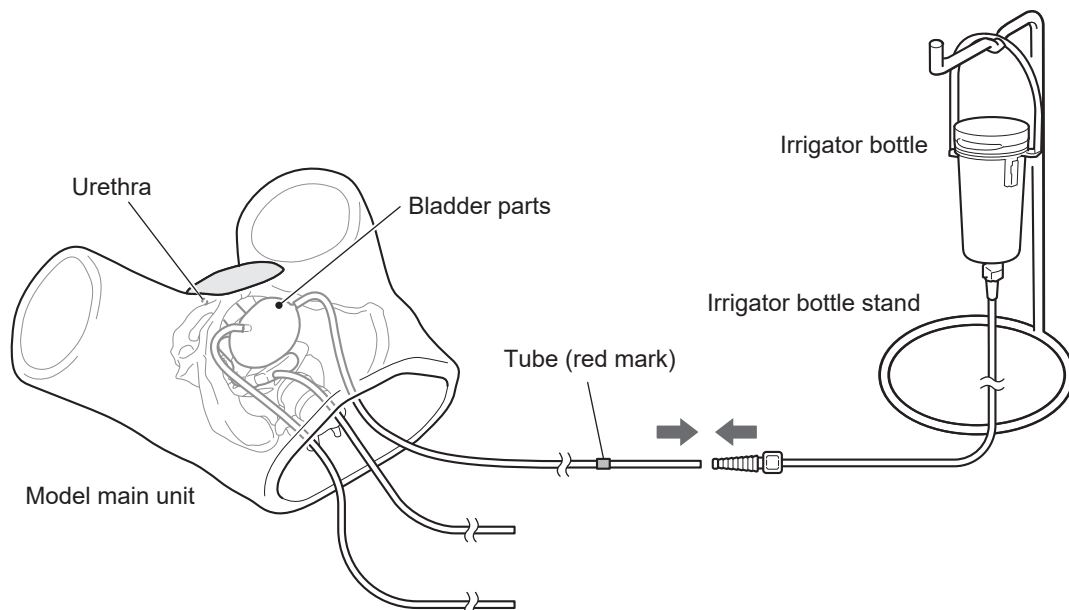


- 2 Hook the tube (unmarked) of the bladder part onto the hook on the side of the irrigator bottle.**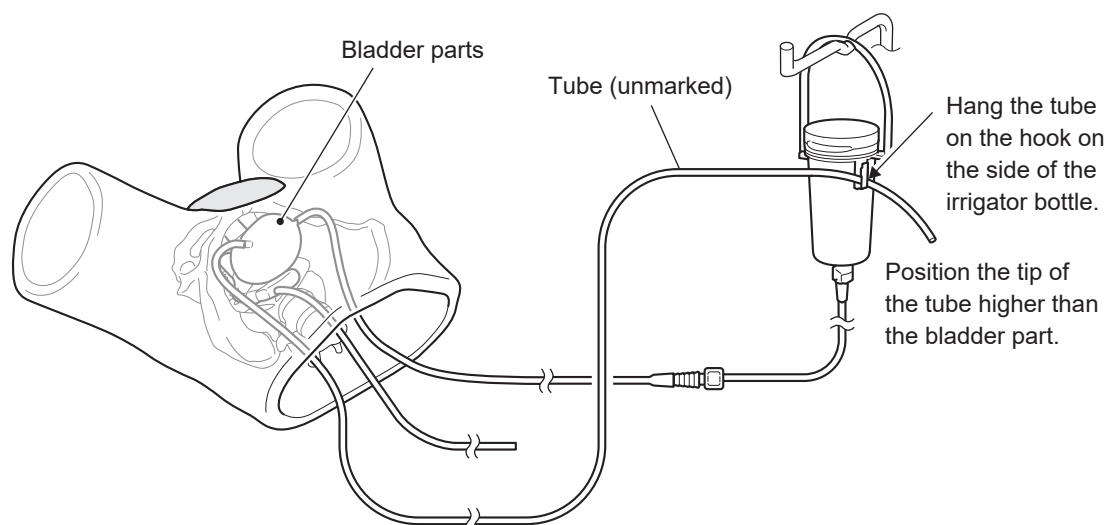

CAUTION

Make sure the tip of the tube (unmarked) is higher than the bladder part. If it is lower, the fluid will be drained from the tube (unmarked) during the training.


3 Open the lid of the irrigator bottle and add at least 500 cc of water.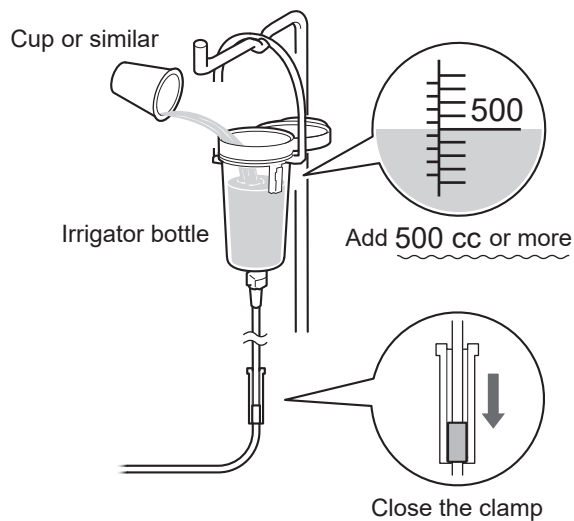

CAUTION

- Close the irrigator bottle clamp before adding water.
- Be sure to use water during the practical training. Using chemicals or other liquids may cause product deterioration.
- Use room temperature tap water. Using hot or boiling water may cause the product to deteriorate.
- If there is less than 500 cc of water, the amount of water that accumulates in the bladder part will be insufficient and water will not be drained properly. Be sure to put at least 500 cc of water in the irrigator bottle.

4 Open the clamp to allow water to flow into the bladder part.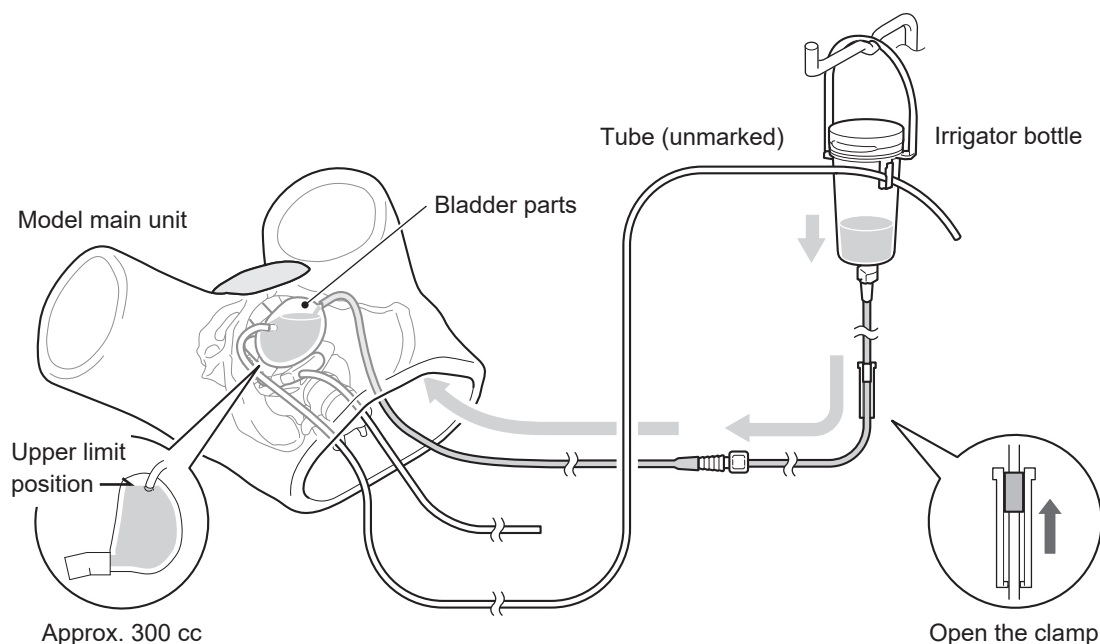

CAUTION

After filling the bladder part with water, please take note of the following:

- Do not pour more than 300 cc of water into the irrigator bottle, as this may cause water to leak.
- Do not lift the model main unit higher than the point where the tube (unmarked) is hooked, as water will be drained through the tube (unmarked).

TIP


- It takes about 2 to 3 minutes for the bladder part to fill with water.
- The bladder part holds approximately 300 cc of water. Water supply stops at the upper limit position.
- Keep the clamp open during catheterization practice.

7 How to use

7-1 Catheterization training

A Nelaton catheter or a balloon catheter will be used for catheterization training. Please prepare a syringe separately for the balloon catheter training.


- 1 The tray is placed on the external genitalia near the urethral opening.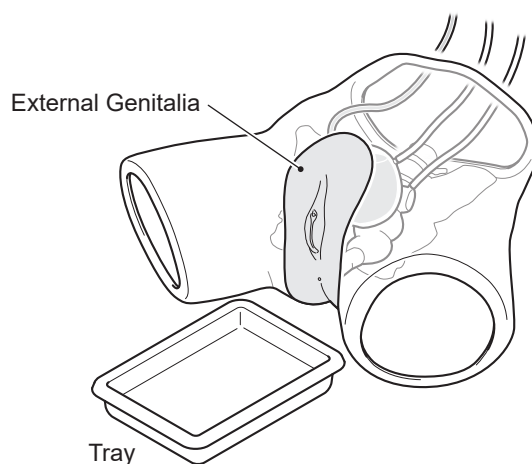

TIP

Please use the included tray or a tray that holds more than 600 cc.

- 2 Apply lubricant oil to the tip of the catheter.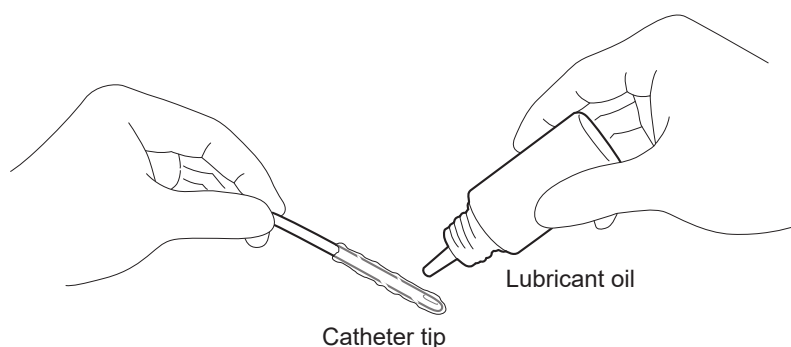


CAUTION

If you insert a catheter through the urethral opening without applying lubricant oil to the tip of the catheter, the urethra will be scratched. Be sure to apply lubricant oil to the tip of the catheter.

TIP

This product is compatible with 12 to 18 Fr catheters.

3 The catheter is inserted through the urethral opening.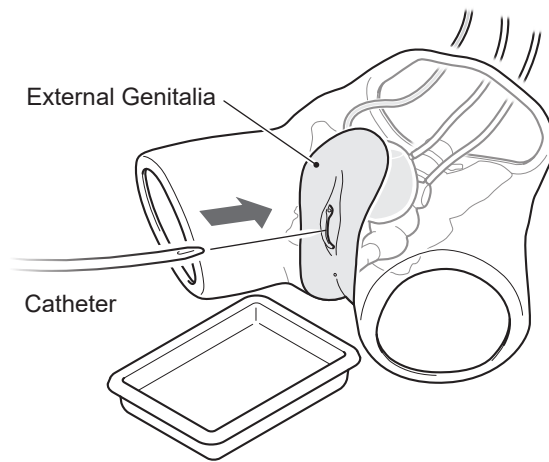


CAUTION

When inserting the catheter, do not force it in as this will damage the urethra.

< Nelaton catheter >

Water will drain from the Nelaton catheter.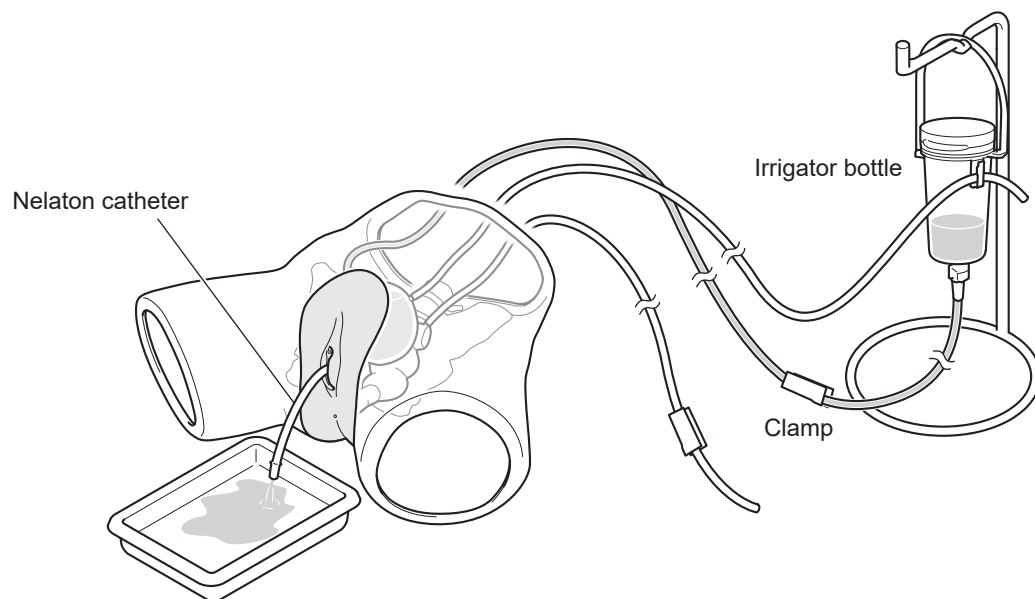


TIP

- To stop water from draining, close the clamp on the irrigator bottle.
- When the water level gets low, add more water to the irrigator bottle as needed.

< Balloon catheter >

The balloon of the balloon catheter is inflated using a syringe, and the balloon catheter is placed in place. ► Water will be expelled from the balloon catheter.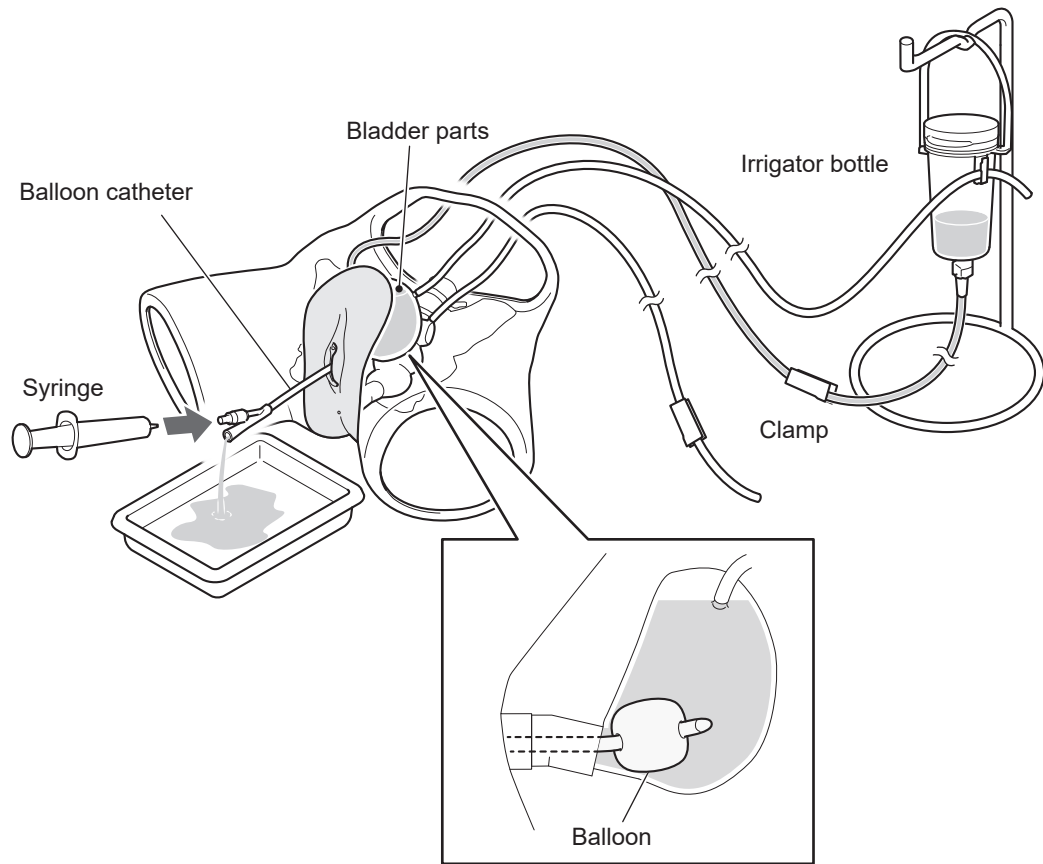

⚠ CAUTION

When removing the balloon catheter, be sure to completely remove all air from the balloon using a syringe.


● TIP

- To stop water from draining, close the clamp on the irrigator bottle.
- When the water level gets low, add more water to the irrigator bottle as needed.

7-2 Enema training

Please prepare an enema separately.


- 1 Place the tray on the end of the rectal part tube.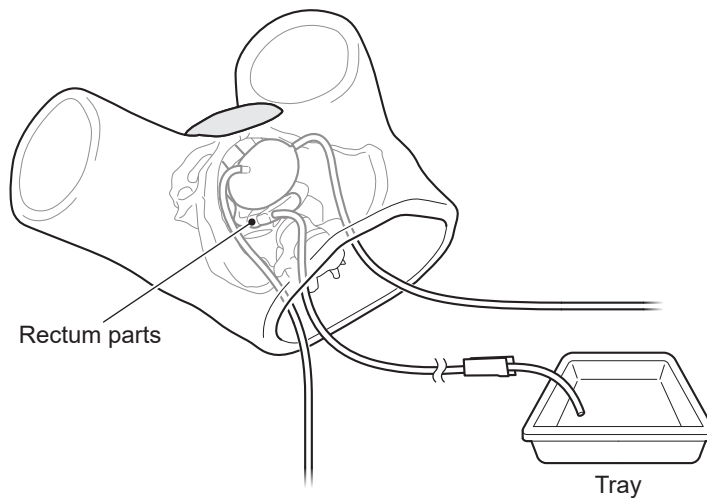

TIP

Please use the included tray or a tray that holds more than 600 cc.

- 2 Apply lubricant oil to the tip of the enema syringe.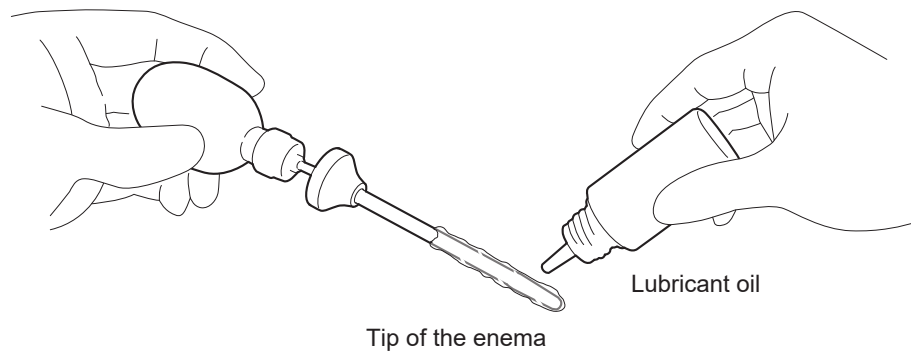


CAUTION

If you insert an enema into the anus without applying lubricant oil to the tip of the enema, it may cause scratches in the anus. Be sure to apply lubricant oil to the tip.

TIP

This product is compatible with enema tubes with outer diameters of 12 to 18 Fr.

3 The clamp is closed and the enema syringe is inserted into the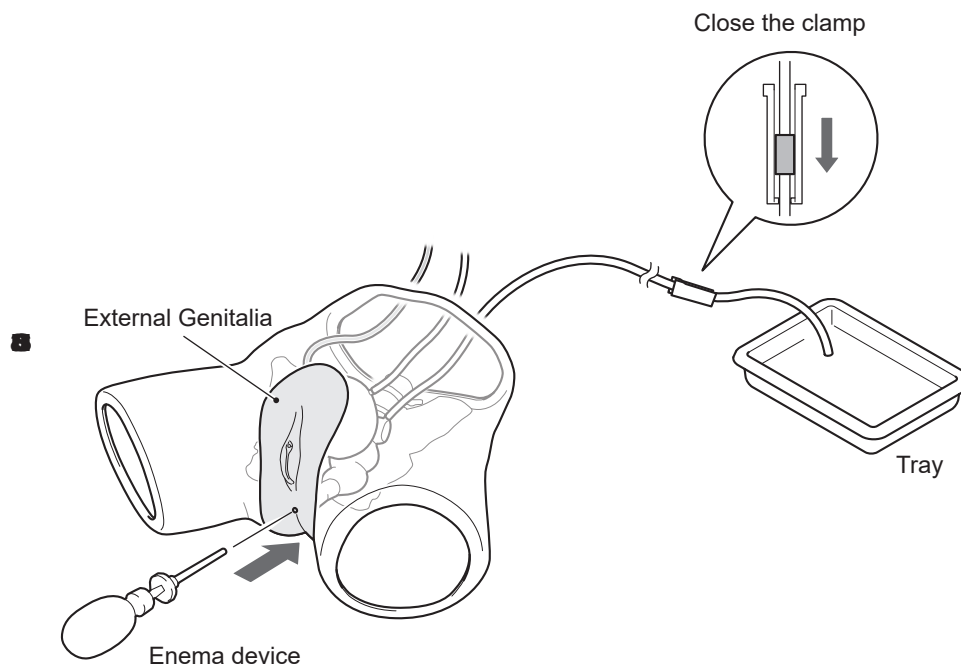

CAUTION

When inserting the enema, do not force it in as this may damage the anal area.

TIP

- When the tip of the enema reaches the inside of the rectum part, water can be injected. The drainage water should be collected in the included tray.
- You can create resistance by adjusting the clamp release.

7-3 Cleansing training

The model main unit itself can be used for genital cleaning practice.

CAUTION


Use a cloth dampened with water to clean. Using anything other than water, such as benzine, thinner, or solvents, may cause the product to deteriorate.

TIP

- After the training, the external genitalia may become sticky or less slippery, but this does not affect the quality. However, if the external genitalia becomes less slippery, it may become more likely to tear, so please handle it with care.
- After the lesson, wipe off any moisture thoroughly and apply the included baby powder.

8 Putting away


- 1 Prepare a tray to collect the drainage and remove the tube (red mark).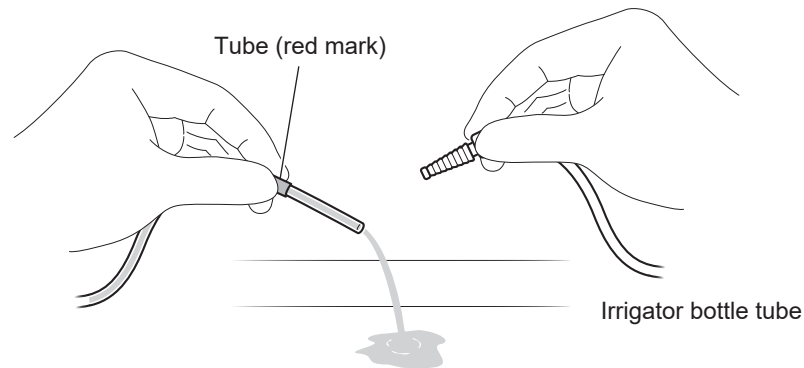


CAUTION

Make sure the irrigator bottle clamp is closed before removing the tube (red mark). If the clamp is open when you remove the tube (red mark), water will drain from the irrigator bottle tube.

- 2 Remove the tube (unmarked) attached to the hook on the side of the irrigator bottle and discard any water accumulated in the tube.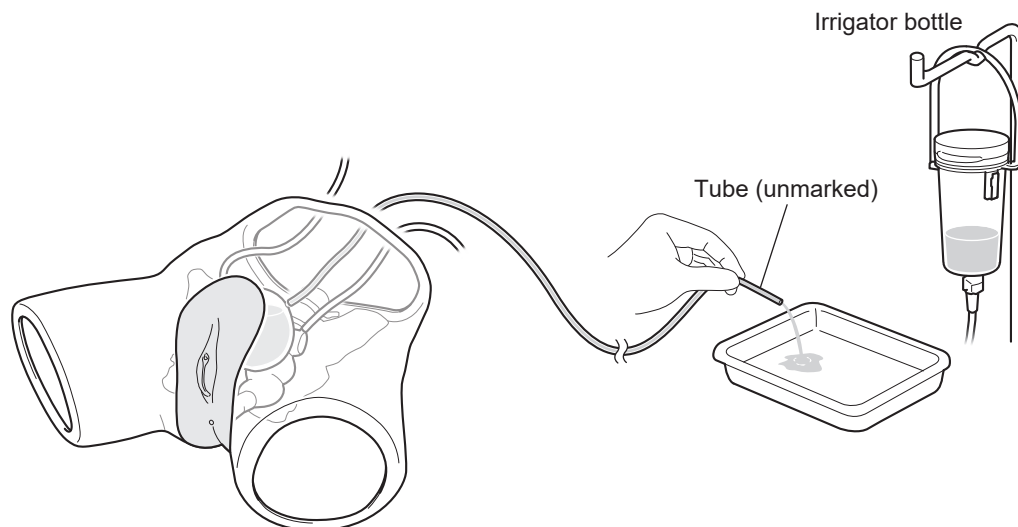


3 Discard any water remaining in the irrigator bottle.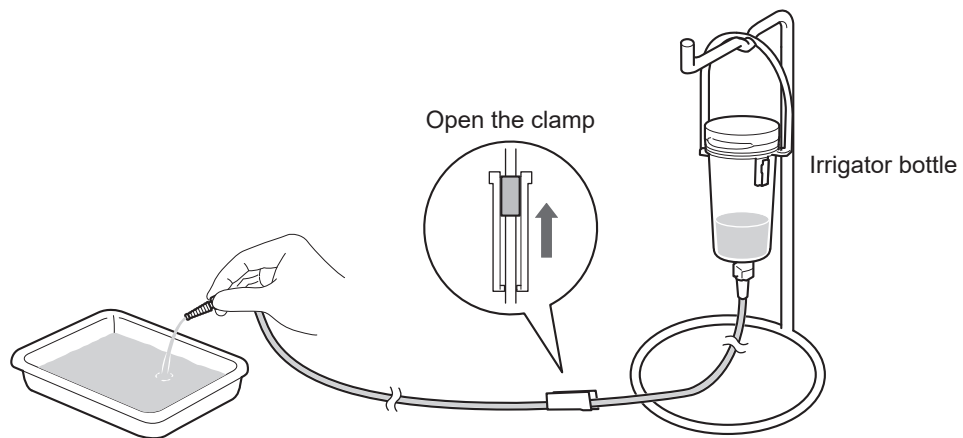


TIP

The water inside the irrigator bottle can also be removed by opening the cap of the irrigator bottle and pouring the water out. In this case, close the clamp of the irrigator bottle.

4 Remove the rectum parts from the model main unit and discard any water remaining in the rectum parts.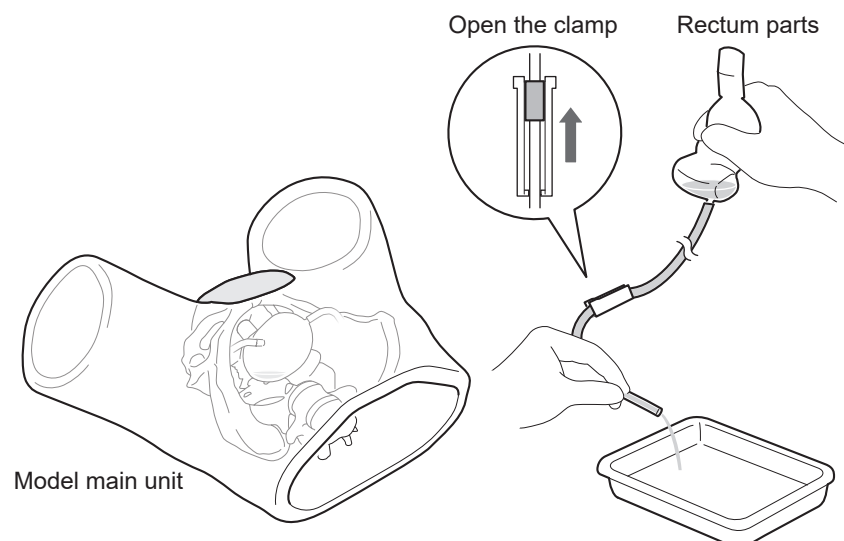


- 5 Remove the bladder parts from the model main unit and discard any water remaining in the bladder parts.**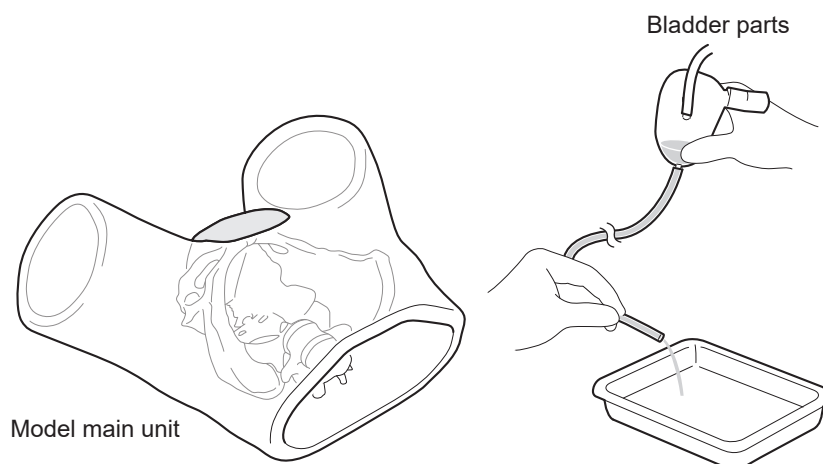

CAUTION

- To remove the bladder part, rotate it counter-clockwise to remove it.
- When removing the bladder part, do not remove the tubes connected to the bladder part as this may cause leakage.

- 6 Apply the included baby powder to the external genitalia.**

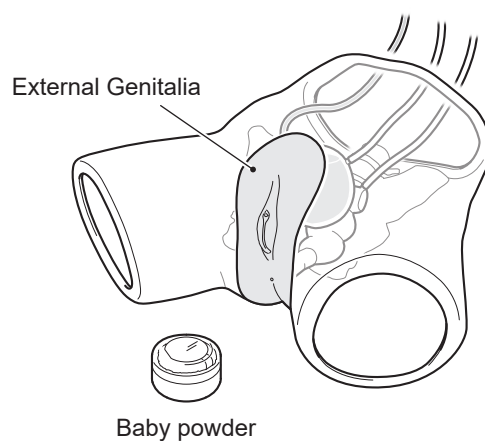TIP

To prevent deterioration of the product, after use, wipe off any moisture thoroughly and apply the included baby powder.

9 Cleaning and storage

Be sure to observe the following when cleaning and storing the product.

■ Once training is finished

After completing the practical training, please observe the following precautions.
(Please also read the “When storing for long periods” section.)

● Please do the following to protect the external genitalia and prevent stickiness.

- If the external genitalia feels sticky, apply the included baby powder.
Be careful not to get baby powder into the urethral opening.
- Apply the included lubricant oil to the urethral opening and around the anus on the external genitalia.

● Please drain the water when not in use.

Leaving water in the bladder or rectum parts can cause an unpleasant odor and deterioration of the product.

● Wipe off any moisture on the product.

Storing the product with moisture remaining on it may cause an unpleasant odor.

■ When storing for long periods

When not using the product for long periods, clean it carefully and store it away while being sure to observe the following.
(Be sure to read the instructions in “Once training is finished” also.)

● Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, discoloration or deterioration may occur.

● Dry all parts before storing.

Dry in the shade for about half a day to a day to completely dry all parts. Storing with moisture remaining on it can cause an unpleasant odor.

● To clean the foreskin, use a soft cloth to gently wipe the surface.

- If it is extremely dirty, wipe it with a cloth which has been moistened with diluted alcohol, and then wipe it again with a dry cloth.
- Do not use benzine, thinner or other solvents. In addition, do not apply alcohol directly to the surface. It may cause the product to deteriorate.
- If you write on the product using a ballpoint pen or similar object, the surface of the product will become smeared with ink which cannot be removed.

● **Store the product in the exclusive case.**

To prevent the product from coming into contact with outside air as much as possible, please use the exclusive case that the product was stored in at the time of purchase it and store the product with the exclusive case securely locked.

10 Specifications

The specifications of the [M212-1] transparent female catheterization and enema model and the [M212-2] transparent male catheterization and enema model are the same.

■ Overall product

Total weight	6 kg (This is the weight of the model main unit and accessories when placed in the exclusive case.)
--------------	--

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SAKAMOTO MODEL C O R P O R A T I O N

Overseas Sales Department
1-9-29, Senbahigashi, Minoh City,
Osaka 562-0035, JAPAN

Tel +81-72-729-8350

Fax +81-72-729-8385

URL <https://www.sakamoto-model.com/>

E-Mail trade@sakamoto-model.com

Product specifications and appearance are subject
to change for improvement without notice.

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- Some of the components used in this product are manufactured by other companies, and product names, etc. for these components may be trademarks or registered trademarks of the manufacturers.