

M106-1 Takeshi



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

- Introduction/Table of Contents 1
- Purpose/Content 2
- Caution on Use 3
- Storage Instructions 4

!
Note

Confirm the main unit and accessories directly after the product arrives. If there is any defect or breakage, inform the dealer or Sakamoto Model.

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “Takeshi.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Cautions on Use.....	3
Storage Instructions.....	4


Purpose

Training can be performed for the following items:

- Posture change
- Whole-body wiping
- Bandaging

Content

1. Main unit
2. Baby powder 1 bottle

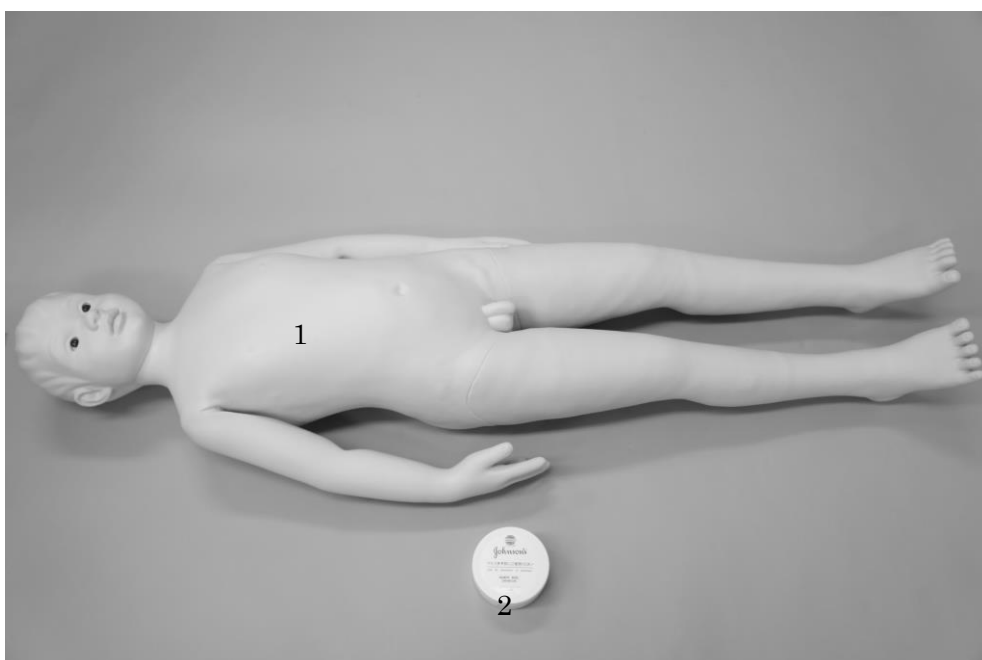

Caution on Use

1. Do not put the model on print including newspapers because it is made of silicon and will easily become dirty or stained. Clean dirt from the skin by wiping it gently using neutral detergents or alcohol-based chemicals.
2. Do not use any permanent markers or inks on the model because they will be absorbed into the skin and become impossible to remove.
3. Carry the model with care. Waterproofing will be lost if it is damaged or torn.
4. Handle the model softly without twisting the skeleton/joints forcibly or moving them beyond the moveable range. Otherwise, it might be damaged or the bonded portion might peel off. Moving a joint forcibly might damage the model.



- 1) Bend or revolve the wrist through the moveable range of the human joint.
Bending or revolving it excessively might lead to breakage of the joint.
- 2) Although the cervical segment can move back and forth as well as from side to side, do not apply a force exceeding the moveable range.
5. If anything strange is detected when this model is being used, stop using it and notify the dealer or us as soon as possible.
6. This product is a simulator for users to perform advanced training. Nonetheless, it does not guarantee that a similar effect can be obtained by actual treatment.
7. Note in advance that this product is subject to change without notice to improve it.
8. If somewhere on the model is sticky, apply the accessory powder lightly.
Using the model without taking the necessary action might lead to breakage

Storage Instructions

1. After training ends, wipe off water well, and add a small quantity of baby powder.
2. When the model becomes dirty, wipe off the dirt using a soft cloth with a neutral detergent.
After that, wipe the model with a cloth moistened with water, and dry it. Then, dust the model with a small quantity of baby powder.
3. Be careful as some silicon chemical may ooze and stain the nightwear if the model is stored without removing it.
4. Lay the model with each joint stretched before storing it.
If there is more than one model, store them one by one without piling them up.
5. Do not store the model anywhere that is exposed to direct sunlight or is hot and humid.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035, JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com