

"Musclemate" 2 - Intramuscular Injection Simulator

● Weight/About 0.5kg ● Case size/W32×D12×H11cm ● Accessories/Drain bag, Belt, Storage case

Mastery of the hard-to-learn feeling of "depth"

- An all-skin model that has achieved unprecedented reality.
- Shows the correct or incorrect injection site including the depth with LED lamps.



Feature

Correct or incorrect is indicated with built-in lamps and a buzzer.



[Green lamp]

OK

The Green lamp is lit when the injection site and the depth are correct.



[Red lamp]

Error regarding the site

The red light is flashed, and the buzzer goes on, when the injection site is incorrect.



[Red and green lamps]

Error regarding the depth

Both the red and green lamps are flashed, and the buzzer goes on, when the injection site is correct but too deep.

* The light of the LED lamps is seen through the skin.

Practice

Preparation



Palpation



Pinching



Brachial I.M. injection



Subcutaneous injection



* Water (tap water only) can be used.