

# M148-3 “Veinmate”2



SAKAMOTO MODEL  
CORPORATION

User's Manual (151218)

## — I N D E X —

●Introduction/Table of Contents	1
●Purpose/Content	2
●Precautions	3
●Usage/Storage method	4

<b>!</b> <b>Note</b>	Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.
-------------------------	--

株式会社坂本モデル



## Introduction

---

Thank you for purchasing our product, “Veinmate2.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

## Table of contents

---


Introduction／Table of Contents.....	1
Purpose／Content.....	2
Precautions concerning handling.....	3
Usage／Storage method.....	4

## Purpose

- 1 Intravenous injection
- 2 Blood sampling

## Content

- 1 Body
- 2 Irrigator
- 3 Wearable belt

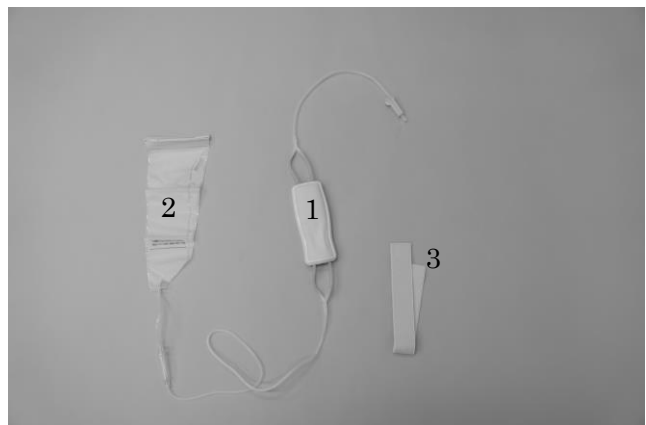

## **Precautions concerning handling**

1. If hanging the blood bag in a higher position or increasing the fluid poured into the bag, water pressure rises and might cause a water leak. Keep the appropriate position and quantity (below the line printed on the blood bag).
2. If coloring water, use the dedicated simulated blood. Not doing so might make the needle mark conspicuous because ink remains inside the needle hole on the epidermis.
3. Be careful to keep the connector (joint) of the tube unplugged while using the product.
4. When using this product to train in injection, use a fresh needle tip. Otherwise, the epidermis might be degraded.
5. Do not dig the injection needle into the product vertically, or use the needle by applying a strong force sufficiently to smash into the base because it is very dangerous.
6. To perform training using this product, the users must be in pairs.
7. If anything strange is detected when this model is being used, stop using it and notify the dealer or us as soon as possible.
8. This product is a simulator for users to perform advanced training. Nonetheless, it does not guarantee that a similar effect can be obtained by actual treatment.
9. Note in advance that this product is subject to change without notice to improve it.

### **\*Pad's life**

Needle marks are left on the vessel tube in the pad on using the tube repeatedly. When fluid starts leaking from the tube a lot, the pad has come to the end of its life. Please purchase a new one.

Since the model is designed so that the tube cannot be replaced, be careful not to pull it out.

## Usage

1. Close the clamp completely.
2. Hang the blood bag (irrigator).
3. As an appropriate height, keep the product so that the irrigator's bottom is approximately 10 cm higher than the pad.
4. Confirm that each joint is connected securely, inject water or a very lightly tinted fluid into the irrigator, to the height of where the arrow mark indicates. "➡"
5. Put the wearable belt on the arm.
6. Put the side equipped with the clamp on the wrist side.
7. Loosen the clamp, add fluid to the front of it, and close it completely.
8. Start training.

## Storage method

1. Do not store the product in a place exposed to direct sunlight or one that is hot and humid.
2. Most of the water injected into the vessel is drained immediately.
3. Store the product after loosening the clamp, draining all the fluid, and drying it.
4. Squeeze the water accumulated in the sponge by pressing the epidermis, and dry the product in the shade.



---

**SAKAMOTO MODEL**  
C O R P O R A T I O N

---

Head office

34,Higashitakagicho,Shimogamo  
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka  
562-0035,JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com