

# M152-2

## Clear 3D Model of Gluteal Injection Site



SAKAMOTO MODEL  
CORPORATION

### User's Manual (151218)

#### — I N D E X —

- Introduction/Table of Contents 1
- Purpose/Content 2
- Cautions on Use 3
- Instructions for Use/Storage Instructions 4

**!**  
**Note**

Confirm the main unit and accessories directly after the product arrives. If there is any defect or breakage, inform the dealer or Sakamoto Model.

株式会社坂本モデル



## Introduction

Thank you for purchasing our product, “Clear 3D Model of Gluteal Injection Site.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

## Table of contents

Introduction / Table of Contents.....	1
Purpose / Content.....	2
Cautions on Use.....	3
Instructions for Use / Storage Instructions.....	4

## **Purpose**

The following three intramuscular gluteus injection sites can be practiced and checked.

- 1 Technique for locating the upper outer quadrant of the buttock
- 2 Clarke's points
- 3 Hochstetter's site

## **Content**

- 1) Training model



## **Cautions on Use**

- 1) Handle this product with care because it may break if dropped or if any excessive load is applied.
- 2) Do not remove the transparent cover.
- 3) Do not touch the structure (bone and muscle) inside the transparent cover. Otherwise, you may damage the product.
- 4) Carefully read the instructions for use when training with injection needles, and do not use it for any other applications.
- 5) Do not remove the description labels on each site.
- 6) If the product gets dirty, use kitchen detergent or soap on a soft cloth to remove the dirt or stain.  
Do not use solvents such as benzene, thinner or polishing powder.
- 7) If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 8) This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee the same results during actual treatment.
- 9) Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.

## **Instructions for Use**

- 1) Take out the training model, and place it on a table with a non-slick surface.
- 2) Use the description labels for each site which are affixed to the transparent cover to confirm the contents.
- 3) When using injection needles for training, there is a hole at the injection site on the transparent cover. Insert the injection needle where the hole is located. In addition, there is a round plastic flap attached above the gluteus medius. An injection needle can be inserted here as well.

## **Storage Instructions**

- 1) Avoid storing the product in direct sunlight or in locations with high temperature and humidity.

---

**SAKAMOTO MODEL**  
C O R P O R A T I O N

---

**Head office**

34,Higashitakagicho,Shimogamo  
Sakyo-ku, KYOTO 606-0865, JAPAN  
Phone: +81-75-701-1135  
Fax: +81-75-722-5638  
E-mail:[info@sakamoto-model.com](mailto:info@sakamoto-model.com)  
URL:<http://sakamoto-model.co.jp>

**Overseas Sales Department**

1-9-29,Senbahigashi,Minoh City,Osaka  
562-0035,JAPAN  
Phone: +81-72-729-8350  
Fax: +81-72-729-8385  
E-mail:[trade@sakamoto-model.com](mailto:trade@sakamoto-model.com)