


SAKAMOTO Gluteus Injection Trainer (lateral recumbent model)

- Weight/About 3.2kg
- Accessories/Cover to check the injection site, Drain bag

A lateral recumbent model appropriate for injections at Clarke's point and Hochstetter's point.

Being a lateral recumbent position, the model permits more practical training.

The epidermis is made of a special material that leaves no needle mark.

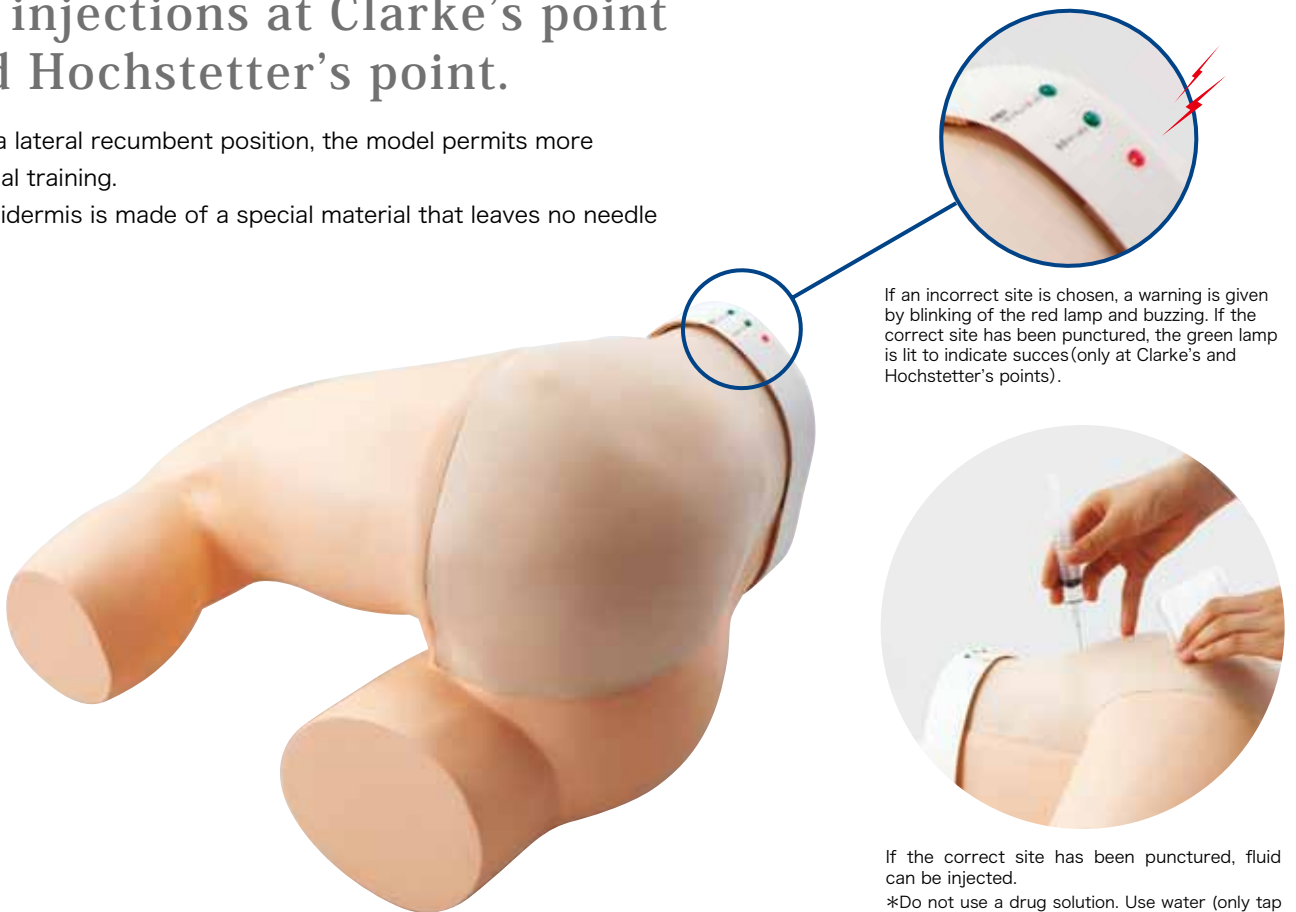

If an incorrect site is chosen, a warning is given by blinking of the red lamp and buzzing. If the correct site has been punctured, the green lamp is lit to indicate success (only at Clarke's and Hochstetter's points).

If the correct site has been punctured, fluid can be injected.
*Do not use a drug solution. Use water (only tap water).

Practice

Hochstetter's point



Clarke's point



The skeletal landmarks such as the anterior superior iliac spine, posterior superior iliac spine, and greater trochanter can be felt, and the model is ideal for practicing the determination of the injection site.

*M152-2, a "Clear 3D model of Gluteal Injection Site", is useful to learn the internal structures.