


M155

Intramuscular Injection Model of Upper Arm Muscles

● Weight/About 9kg ● Case size/W77×D58×H28cm ● Accessories/Drain bag, Storage case

Can choose injection spots accurately, practical training available.

- When an injection is made in a wrong spot or if it's made too deep, a red lamp light up and a buzzer sounds a warning as well. Also a lamp in the nerves will go on and off. When an injection is made safely, a green lamp will notify that.
- The inner structure of nerves and veins in the upper arm is shown so one can make an accurate choice of injection spots, avoid the mistake of making injections to the nerves and veins, and practice in a more real and practical way.
- The model ranges from the shoulder to the elbow thus making it possible to practice intramuscular and hypodermic injections in 2 different spots.
- Water (only tap water) can be infused.

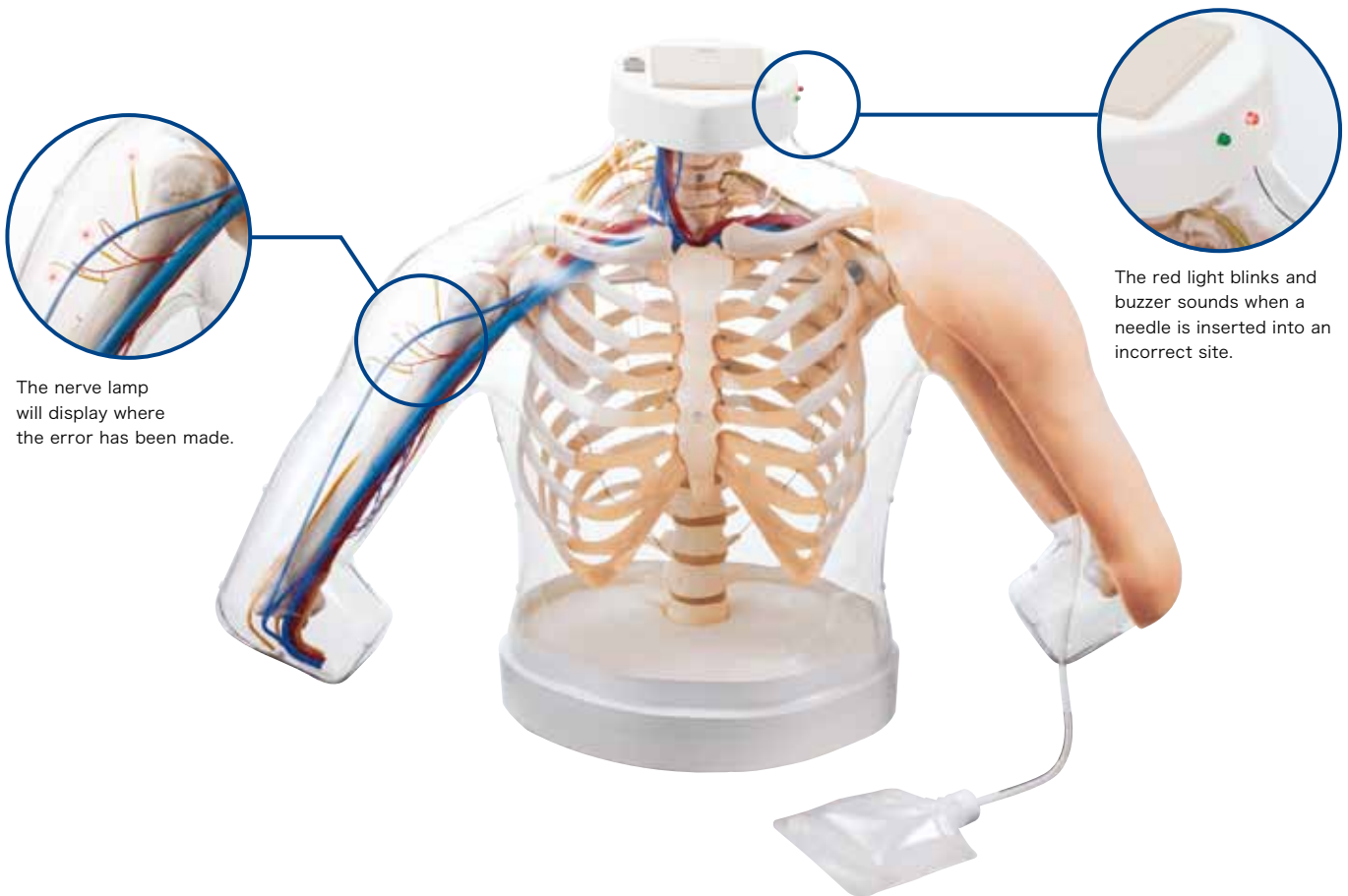

The nerve lamp will display where the error has been made.

The red light blinks and buzzer sounds when a needle is inserted into an incorrect site.

Practice



Can practice choosing injection spots correctly.



Can confirm injection sites in upper arm by muscles palpation of the acromion apophysis.