

M163 Swallowing Mechanism Model



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

●Introduction/Table of Contents	1
●Purpose/ Content	2
●Precautions concerning handling	3
●Usage	4
●Storage method	5

! Note

Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “Swallowing Mechanism Model.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Precautions concerning handling.....	3
Usage.....	4
Storage method.....	5


Purpose

The following can be learned.

1. Swallowing mechanism
2. Oral care

Content

1. Main unit
2. Transparent cover sheet with anatomical indications
3. Pillow
4. Food bolus model (with a pointer)
5. Baby powder

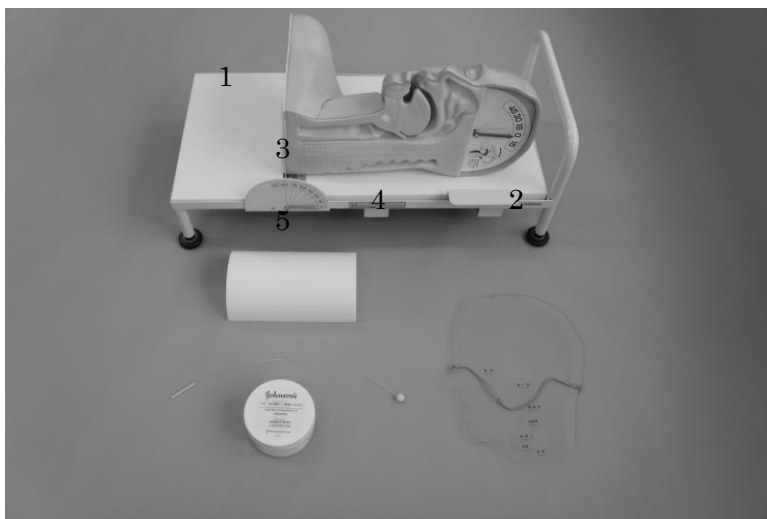

Precaution concerning handling

1. Avoid prolonged contact with resin products, as it may cause distortion of the face and neck parts of the product.
2. Due to the nature of the material, the face and neck parts are likely to get dirty easily. Note that stains of a ballpoint pen or marker on the product cannot be removed.
3. Light dirt on the face and neck parts can be washed with neutral detergent.
(Do not use solvents, such as thinner, because they melt the resin parts.)
After cleaning, wipe off all remaining water and apply baby powder.
4. Falls or an excessive load may cause damage to the product.
5. Hold both sides of the main unit when moving the bed.
Adjust the angle of the head and bed to 0°, and lay down the head part before moving it.
Please refer to the operating instructions for detailed information.
6. When raising the head of the bed, hold the back of the head with one hand and the chest with the other. Do not hold soft parts of the face or neck.
Please refer to the operating instructions for detailed information.
7. Before bending the neck, the head must be raised. Tilt the neck while holding the back of the head with one hand and the chest with the other. Do not touch the head angle meter. Do not hold soft parts of the face or neck.
Please refer to the operating instructions for detailed information.
8. The head must be raised when tilting the bed. Do not touch the bed angle meter.
Please refer to the operating instructions for detailed information.
9. Avoid moving the food bolus excessively in the mouth, esophagus, and trachea as this may cause a tear in the face.

10. If you notice any defect in the product, discontinue use immediately and contact our company or the distributor.
11. This product was designed to simulate advanced medical training. However, this does not guarantee similar results in actual treatment.
12. The specifications of this product may be upgraded without prior notice.

Usage

1. Take the main unit out of the hard case, and place it on a non-slip desk surface. Make sure you hold both ends of the bed, when taking the main unit out. (Do not try to move it by holding one side.)



2. Remove the tape stabilizing the head part, and take the head out of the bag.

3. Raise the head part. When doing this, hold the back of the head with one hand and the chest with the other. Do not hold soft parts of the face or neck.



4. The neck can be tilted approximately 15° forward and 45° backward. Bend the neck by holding the back of the head with one hand and the chin with the other. The head must be raised before bending the neck. Do not touch the head angle meter.




5. The head of the bed can be adjusted to 13 angles from the flat position.

You can change the angle by pulling out the stopper from under the movable side of the bed and inserting it in an arbitrary groove.

Before lifting the bed, the head must be raised.

Do not touch the bed angle meter.

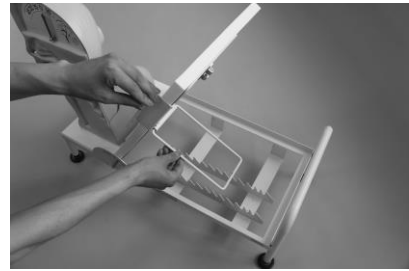

6. Lay the head part down on the bed, and you are ready to start practicing.

7. To move the bed on the table, adjust the angle of the head part and the bed to 0°, and lay the head part down on the bed.

8. The transparent cover sheet can be used by attaching a rubber band to the head part.



9. The pillow and food bolus model can also be used.

Caution must be taken when moving the head part.

Avoid moving the food bolus excessively in the mouth, esophagus, and trachea.

Storage method

1. Dirt on the face or neck parts must be wiped off lightly using a soft cloth with neutral detergent. After cleaning, wipe off with a cloth moistened with water, and apply baby powder.
2. Adjust the angle of the head and bed to 0°, and lay down the head part before storing it.
3. The transparent cover sheet must be removed for storage.
4. Put in the dedicated hard case, and avoid storing the product in direct sunlight or in locations with high temperature or humidity.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035, JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com