

M167

Sakamoto Airway Management Trainer



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

● Introduction / Table of Contents	1
● Purpose / Content	2
● Precautions concerning handling	3
● Usage / Storage method	4

! Note	Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.
-------------------------	--

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “Sakamoto Airway Management Trainer.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Precautions concerning handling.....	3
Storage method.....	4

Purpose

Intubation training is as follows:

1. Oral trachea intubation
2. Nasal trachea intubation
3. Confirmation of single-lung ventilation
4. Confirmation of stomach distention

Contents

- | | |
|-----------------------------------|----------|
| 1. Main unit (external genitalia) | 1 piece |
| 2. The replacement anterior teeth | 3 pieces |
| 3. Silicon oil | 1 bottle |
| 4. Baby powder | 1 bottle |



Precautions concerning handling

1. If resin products contact the head for a prolonged time, the head might be deformed. Treat the product with sufficient care. In addition, do not remove the epidermis of the head.
2. Due to the nature of the material, the face parts are likely to get dirty easily. Note that stains of a ballpoint pen or marker on the product cannot be removed.
3. Light dirt on the face parts can be washed with neutral detergent.
(Do not use solvents, such as thinner, because they melt the resin parts.)
After cleaning, wipe off all remaining water and apply baby powder.
4. The anterior teeth will be broken if strongly impacted.
In the case that a tooth drops in the mouth, pick it out with tweezers.
5. Apply the accessory silicon oil to this product before using tubes.
6. If anything strange is detected when this model is being used, stop using it and notify us as soon as possible.
7. This product is a simulator for users to perform advanced training. Nonetheless, it does not guarantee that a similar effect can be obtained by actual treatment.
8. Note in advance that this product is subject to change without notice to improve it.

Usage

1. Pick the main unit out of the storage case, and put it in a place where it cannot slip, for example, a desktop.
2. It is recommended to use rubber gloves to prevent the dirty.
3. Apply the dedicated lubricant oil to the catheter tip before inserting it.
4. Start the intubation training.
5. The anterior teeth are designed to be broken if the procedure is not performed correctly.

When those teeth are damaged, replace them with the reserve ones.

6. If intubation is correct, both the lungs swell. Otherwise, only a single lung swells.
If the tube is inserted into the esophagus, the stomach swells.

Storage method

1. Light dirt on the face parts can be washed with neutral detergent.
(Do not use solvents, such as thinner, because they melt the resin parts.)
After cleaning, wipe off all remaining water and apply baby powder.
2. Do not store the product in a place exposed to direct sunlight or one that becomes hot and humid. Otherwise, it might be deformed or altered.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035, JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com