

M172 Sakamoto Tracheotomy Trainer



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

● Introduction/Table of Contents	1
● Purpose/Content	2
● Precautions Concerning Handling	3
● Usage/Storage method	4

! Note

Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “Sakamoto Tracheotomy Trainer.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Precautions concerning handling.....	3
Usage/Storage method.....	4

Purpose

Intubation training is as follows:

1. Conventional methods
(Longitudinal, transverse, crucial, U-shaped, and inverted U-shaped incision)
2. Percutaneous tracheostomy
3. Puncture and incision of the thyrohyoid ligament

Contents

- 1 Main unit (external genitalia) 1 piece
- 2 Cervical part (with arteries)
- 3 Thyroid cartilage (with an annular cartilage)
- 4 Trachea (4 pieces)
- 5 Epidermis (4 sheets)
- 6 Silicone oil



Precautions concerning handling

1. Do not set the cervical part on the main unit without setting the thyroid cartilage and trachea on it. (Otherwise, the cervical part might be torn.)
2. When the model gets dirty, wipe off the dirt using a soft cloth with a neutral detergent.
* Do not use benzine, thinners, or polishing powders.
3. If anything strange is detected when this model is being used, stop using it and notify us as soon as possible.
4. This product is a simulator for users to perform advanced training. Nonetheless, it does not guarantee that a similar effect can be obtained by actual treatment.
5. Note in advance that this product is subject to change without notice to improve it.

Usage

1. Pick the main unit out of the storage case, and put it in a place where it cannot slip, for example, a desktop.
2. Insert a trachea into the thyroid cartilage. Set the trachea with the dent upward.
(The tracheas are usable more than once.)
3. Set the trachea so that it is fitted into the dent of the cervical part.
4. Wind the epidermis around the cervical part, and close the fastener.
The unused portion is usable by shifting the position.
(The epidermis is usable more than once by shifting left and right or up and down.)
5. Put the cervical part on which the epidermis is wound from the back of the main unit, and set it on a flat surface. The cervical part is designed so that it can fit a bit tightly to ensure good fitting. Therefore, this part can be secured by pushing the main unit a little after placing it.
6. Apply the attached silicon oil before inserting the cannula.
7. Start training.
(Inside the trachea into which cannulae and so on are inserted can be confirmed from the head.)

Storage method

1. If resin products contact the head for a prolonged time, the head might be deformed. Treat the product with sufficient care. In addition, do not remove the epidermis of the head.
2. Do not store the product in a place exposed to direct sunlight or one that becomes hot and humid. Otherwise, it might be deformed or altered.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035,JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com