

M192

Enema Administration Simulator 2



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

| | |
|---------------------------------|-----|
| ●Introduction/Table of Contents | 1 |
| ●Purpose/Content | 2 |
| ●Cautions concerning handling | 3-4 |
| ●Storage | 4 |
| ●Usage | 5 |

!
Note

Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “Enema Administration Simulator 2.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

| | |
|---------------------------------------|-----|
| Introduction / Table of Contents..... | 1 |
| Purpose / Content..... | 2 |
| Cautions concerning handling..... | 3-4 |
| Storage..... | 4 |
| Usage..... | 5 |


Purpose

Training can be performed for the following items:

- 1 Enema Training
- 2 Sponge Bath Training

Contents

1. Main Unit 1 piece
2. Wearable waterproof pants 1 piece
3. Dedicated stand 1 piece
4. Baby Powder 1 bottle
5. Silicone oil 1 bottle
6. Simulated loose excrement (yellowish brown) 1 bottle
7. Container for mixing simulated loose excrement 1 piece
8. Muddler for mixing simulated loose excrement 1 piece
9. Tray 1 piece

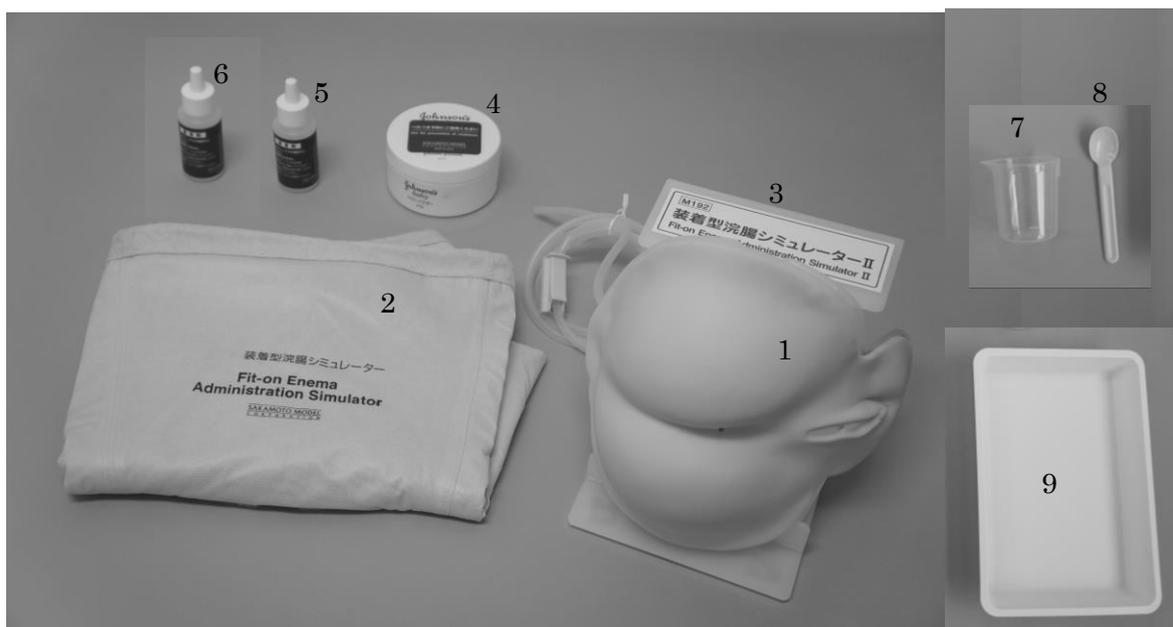

Cautions concerning handling

1. The periphery of the anus on the main unit easily becomes dirty due to the material characteristics, so put on rubber gloves to perform training.
2. Although the wearable pants are waterproof, they are not perfectly water resistant. Pay sufficient attention to ensure durability.
3. When performing training, be sure to apply the accessory dedicated silicon oil before inserting it.
Use the attached dedicated silicon oil only as lubricant oil.
4. Do not use any permanent markers or inks on the model because they will be absorbed into the skin and become impossible to remove.
5. Main unit is removable from the pants. Fit to the magic tape when reattach it.



6. When fitting the simulator onto the dedicated mount, align the back of the simulator (black tube side) with the “hose connection side.” In addition, when removing the simulator from the dedicated mount, disconnect the simulator from the groove located on top of the mount.



7. The soft buttocks on the front of the simulator and the Velcro attachment on the back of the simulator cannot be separated.

Do not try to peel it off because it will damage the simulator.



8. Wrap it in newspaper, etc., and dispose of it with regular household waste or burnable waste.
9. Appropriate size of enemas is 12Fr to 18Fr (4mm to 6mm)
If anything strange is detected when this model is being used, stop using it and notify us as soon as possible.
10. This product is a simulator for users to perform advanced training. Nonetheless, it does not guarantee that a similar effect can be obtained by actual treatment.
11. Note in advance that this product is subject to change without notice to improve it.

Storage

1. Do not store the product in a place exposed to direct sunlight or one that becomes hot and humid. Otherwise, it might be deformed or altered.
2. Wrap it in newspaper, etc., and dispose of it with regular household waste or burnable waste.
3. If the model becomes sticky, apply the attached powder lightly when using the model, as it might lead to breakage.

Usage

1. Bring the main unit prepare it. (You can use dedicate stand or wearable pants.)
2. Apply an adequate quantity of silicon oil to the enema tool before starting training.
3. Appropriate size of enemas is 12Fr to 18Fr (4mm to 6mm)
Use of enemas that exceeded this size cause of failure of models
4. The periphery of the anus on the main unit easily becomes dirty due to the material characteristics, so put rubber gloves on to perform training.
5. Set up a large bucket or similar receptacle because fluid will discharge from the end of the tube.
6. If the product is put on the buddy for performing training, be careful of the people around the model.
7. Resistance to the enema fluid being injected can be adjusted by opening/closing the clamp.
8. How to make simulated loose excrement.
Put 50 CC of water into the instrument to stir and simulate soft stool. Stir the powder into the water for soft stool simulation, and use the muddler to knead the soft stool for 30 seconds. Thereafter, add a little bit of powder at a time until reaching the desired hardness.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail: info@sakamoto-model.com

URL: <http://sakamoto-model.co.jp>

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035, JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail: trade@sakamoto-model.com