


Acupuncture Training Pad

(With indicator light and chime)

● Weight/About 0.3kg ● Case size/W22×D12×H8cm ● Accessories/Belt, Elastic band, Storage case

Two target points of different depth facilitate a practice by level of difficulty

This product is equipped with a sensor that accurately detects the arrival of the target, and indicates whether it is correct or incorrect with a lamp and a chime sound. Accumulating successful experiences also has the effect of increasing confidence.

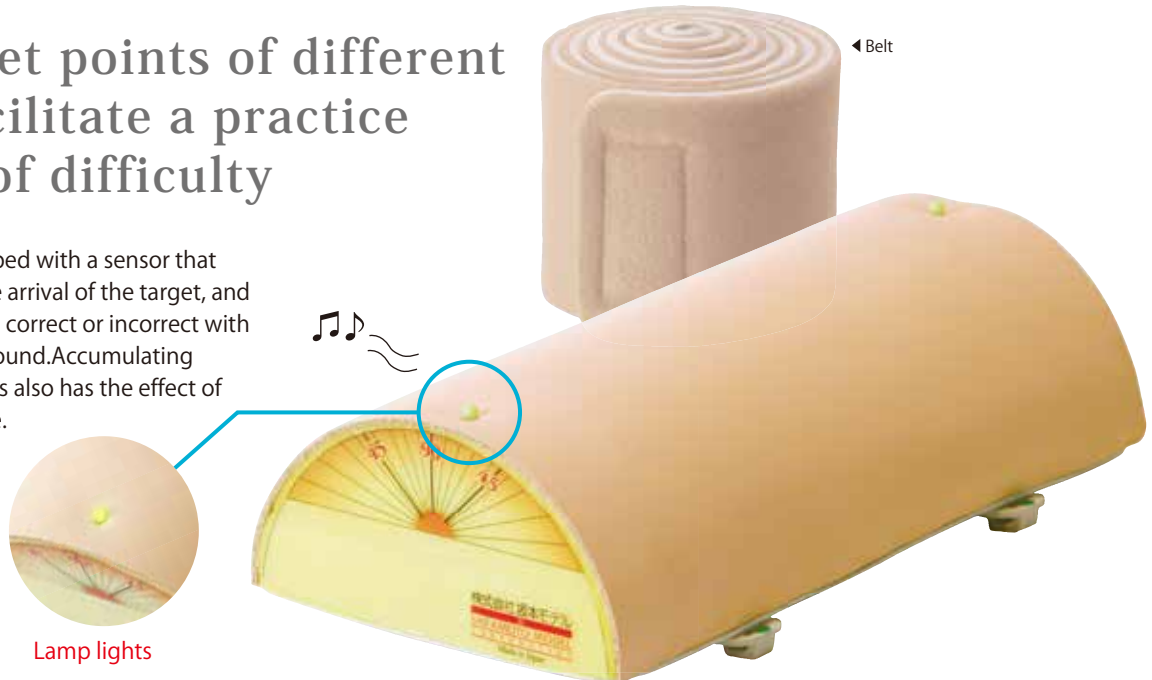

Practice

1 Practice by level of difficulty


Two target points of different depth allows to practice by difficulty levels. The training pad has indicator light and chime sound to inform whether the point of insertion is correct or incorrect, it gives a sense of three dimensional imaging.

● Insertion to the shallow target point



Shallow target point

Deep target point

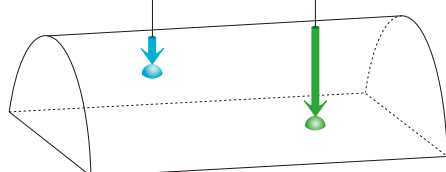

● Insertion to the deep target point



◀ Attach the rubber band to the hook on the side of the pad, this confine the area, and makes it easier to locate the target point.

● Indicator light and chime sound

The green light turns on and a chime sounds when the needle is inserted to the target point. (The light does not turn on when inserted to the wrong point) The shape of sensor which detects the needle insertion is hemispherical, and it judges accurately whether insertion is perpendicular or oblique.

2 Practice by attaching the pad

Hook and loop fasteners can be attached to any part of the body. Pair practice facilitates a realistic needle insertion practice in actual posture. More realistic practice is possible to the moving subject.



<Sold separately Arm for Simulator (M182)>