

Fit-on & Free-standing Types

Female Catheterization & Enema Simulator WECS

Male Catheterization & Enema Simulator WECS

- Weight/About 2kg ● Case size/W58×D46×H29cm
- Accessories/Simulated soft stools (yellowish-brown), Simulated soft stool mixing container, Simulated soft stool stirring rod, Tray, Exchangeable return valve (×2), Bottle, Bottle stand, Balloon catheter 16Fr, Nelaton catheter 8EG, Lubricant oil, Baby powder, Waterproof pants, Exclusive stand, Storage case, Foreskin (men only)

Two training tasks of catheterization and enema practice can be carried out using a single model

In addition, practical training while actually wearing can also be carried out



▲ Female Catheterization & Enema Simulator



▲ Male Catheterization & Enema Simulator



When installed
(using a special stand)



When worn
(using waterproof pants)



When installed
(using a special stand)



When worn
(using waterproof pants)

1 Fully compatible with a variety of needs!

Genital parts can be easily attached and removed at a single touch. To meet a variety of needs, they can be attached to the special stand for training in confirming technique, or they can be fitted to the accessory pants for clinical simulations. The male and female genitals can be switched during training.



2 "2 in 1" catheterization technique

The features of both the previous free-standing type and fit-on type have been combined into a single model. Of course the same feeling of fitting and performance as previous models are achieved.



Balloon catheter insertion can also be checked during free-standing training.



Catheter insertion and urination can also be checked during fit-on training.

3 Enema/cleaning

Enema procedures, not previously available with the fit-on type, have been newly added. In addition, cleaning training using the accessory simulated soft stools and genital cleaning can also be carried out.



Enema training can be performed in the lateral position after fitting.



Cleaning training using the accessory simulated soft stools.

M200-8

Fit-on & Free-standing Types Male&Female Catheterization & Enema Simulator Plus SET (Male/female genitalia set)

- Includes/Body ×1, Male genitals ×1, Female genitals ×1
- Accessories/Simulated soft stools (yellowish-brown), Simulated soft stool mixing container, Simulated soft stool stirring rod, Tray, Exchangeable return valve (×2), Bottle, Bottle stand, Balloon catheter 16Fr, Nelaton catheter 8EG, Lubricant oil, Baby powder, Storage case, Foreskin

