

Hemiplegia Simulation Suit

M165-1 M size

M165-2 L size

- Weight/About 3kg
- Accessories/Shoes, Ankle equipment, Knee equipment, Wrist equipment, Cane, Storage bag

Can realistically learn the inconvenience of hemiplegia only understandable through experience

The "Hemiplegia Experience Suit" allows nursing and care workers to learn the details of hemiplegia caused by cerebrovascular accidents, brain trauma, brain tumors, etc. through a simulated experience of hemiplegia. It was created with the purpose of helping children learn about compassion and consideration.



Features

- Can experience both right or left hemiplegia.
<the sleeve / equipment (ankle / knee / wrist) can be attached to either right or left side.>
- Each equipment is fixed so that one can experience a similar condition to having stiff joints due to hemiplegia, but by using the rubber belt, it is possible to adjust tightness. (So that one can cope with an unexpected occurrence while experimenting, we have left some room.)
- The cane is useful for various movements, and the necessity of it will be strongly felt.
- The shoes were actually designed / developed for the aged and physically disabled. We have adopted these light and hard-to-slip shoes.
- It comes with a bag and is compact, so carrying and storing is easy.
- The suit (jacket) is washable and therefore clean.

Walking

- The irritation and fear of not being able to cross a pedestrians' crossing quickly.
- The necessity of a cane when walking.
- The fear of being in a crowd.



Climbing

- The difficulty of climbing up and down steps.
- The difficulty and anxiety of getting on and off buses / trains / cars.
- The difficulty of standing up and sitting down when using the toilet etc.



Cane



Jacket



Sleeve (for both right and left)



Storage bag

Ankle equipment
(for both right and left)

Shoes

Wrist equipment
(for both right and left)Knee equipment
(for both right and left)

※Sleeve equipment (for ankle, knee, wrist) are attachable to either right or left side.

Specifications

M size	Recommended stature	155cm-165cm
	Shoe size	22.5cm-24.5cm
	Body color	Pink

L size	Recommended stature	165cm~175cm
	Shoe size	25cm-27cm
	Body color	Blue